


HEAD RESTS




TRC0815 - HEADREST SELECTION


REVISION: 05/24/2007

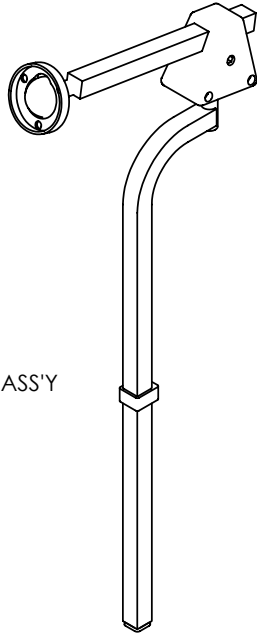


TRC0810
STANDARD HEADREST ASS'Y

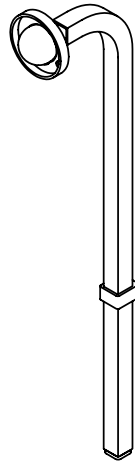

TRC0811
ONYX HEADREST ASS'Y

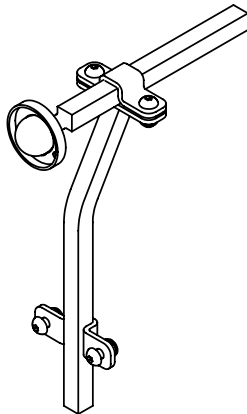

TRC0813
BASIC HEADREST ASS'Y


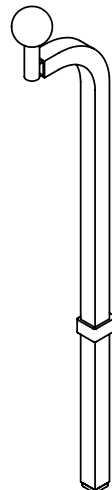


TRC0812
STEALTH HEADREST ASS'Y


**SEE HEADREST ASS'Y
FOR HEAD PAD CONFIGURATIONS**

TRC0810 - HEADREST ASS'Y

REVISION: 05/04/2007

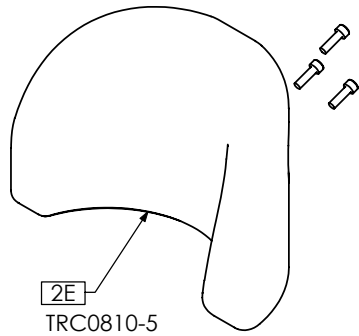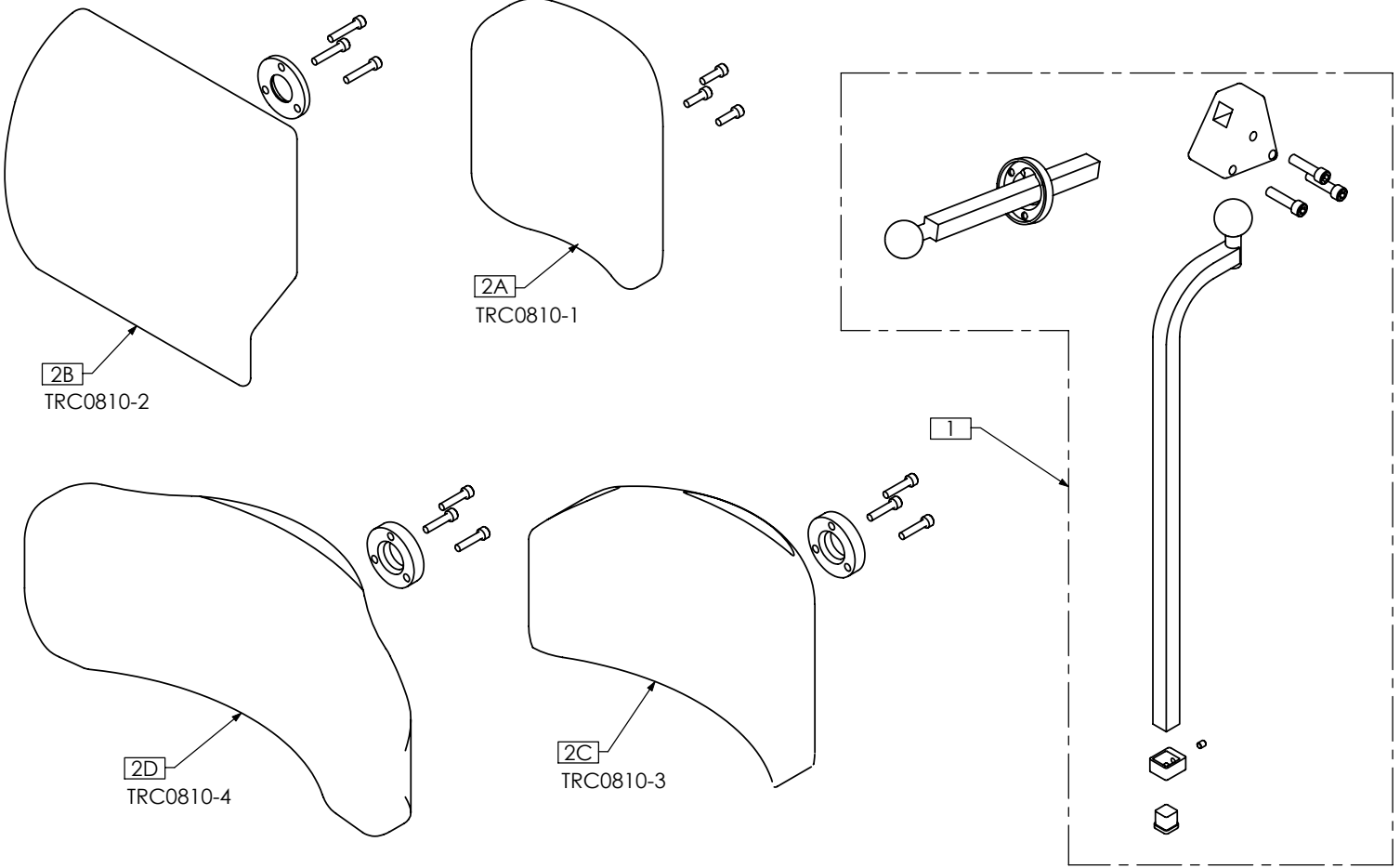

2E
TRC0810-5

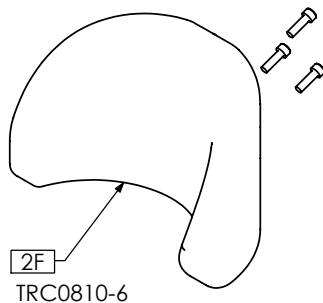


2F
TRC0810-6

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRC0108	HEADREST HARDWARE ASS'Y
	3	TRH0335	1/4-20 x 1" SOCKET HEAD CAP SCREW GR8
	1	TRM0038	LARGE VERTICAL HEADREST ROD
	1	TRM0039	TRIANGULAR MULTI-OFFSET
	1	TRM0037	SMALL HORIZONTAL HEADREST ROD
	1	TRM0040	HEADREST MOUNTING RING
	1	TRM0414	D-RING
2A	1	TRM0005	HEADREST
	3	TRH0585	#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
2B	1	TRM0368	HEADREST, ONYX
	3	TRH0329	#10-32 x 1" SOCKET HEAD CAP SCREW GR8
	1	TRX1541	MOUNTING PLATE
2C	1	TRM0499	HEADREST, STEALTH, 10"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
2D	1	TRM0498	HEADREST, STEALTH, 14"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
2E	1	TRM0282	HEADREST, OTTOBOCK - LARGE
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
2F	1	TRM0283	HEADREST, OTTOBOCK - SMALL
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8

TRC0811 - HEADREST ASS'Y, ONYX


REVISION: 05/04/2007

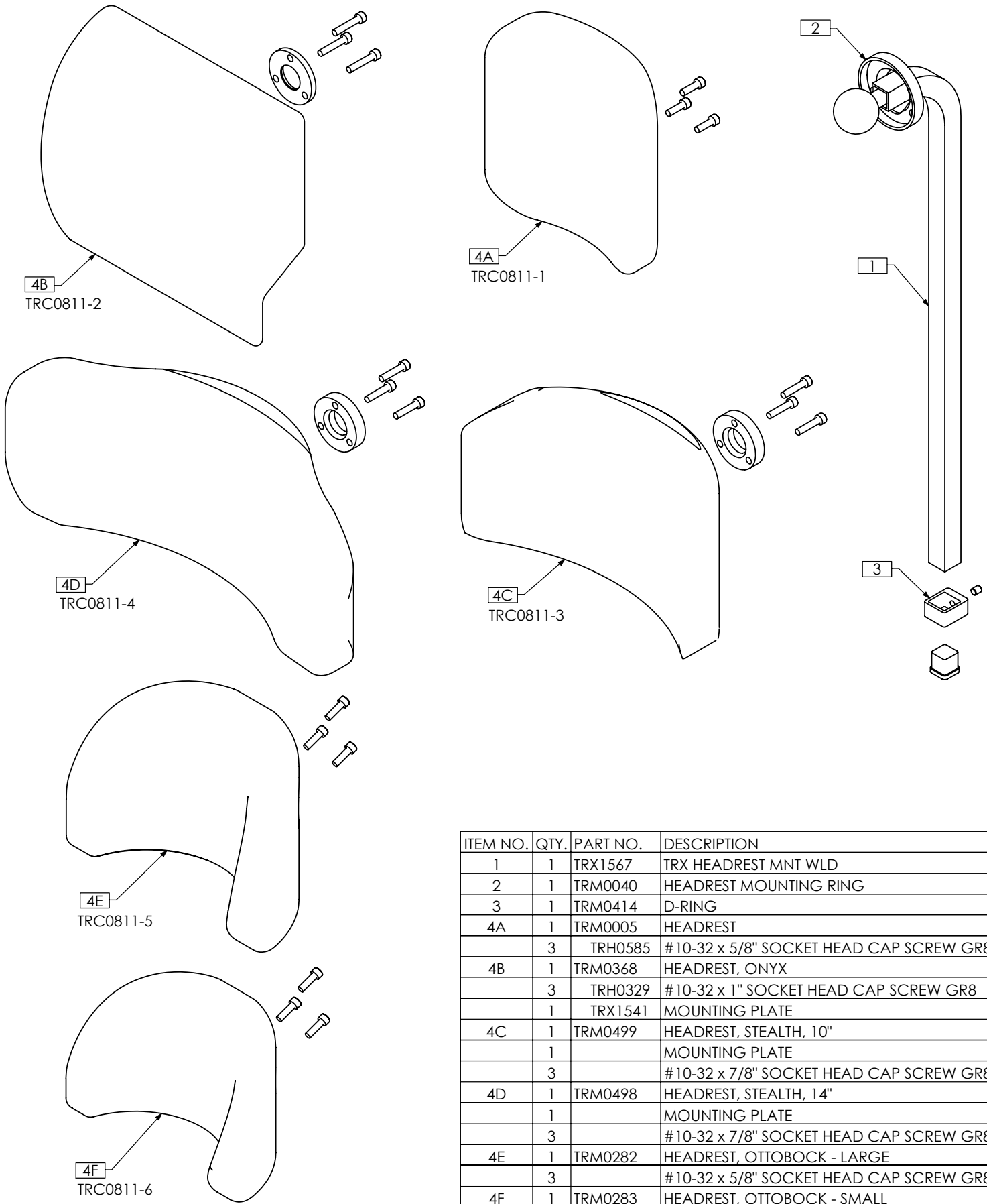

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX1567	TRX HEADREST MNT WLD
2	1	TRM0040	HEADREST MOUNTING RING
3	1	TRM0414	D-RING
4A	1	TRM0005	HEADREST
	3	TRH0585	#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
4B	1	TRM0368	HEADREST, ONYX
	3	TRH0329	#10-32 x 1" SOCKET HEAD CAP SCREW GR8
	1	TRX1541	MOUNTING PLATE
4C	1	TRM0499	HEADREST, STEALTH, 10"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
4D	1	TRM0498	HEADREST, STEALTH, 14"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
4E	1	TRM0282	HEADREST, OTTOBOCK - LARGE
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
4F	1	TRM0283	HEADREST, OTTOBOCK - SMALL
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8

TRC0812 - HEADREST ASS'Y, STEALTH


REVISION: 05/04/2007

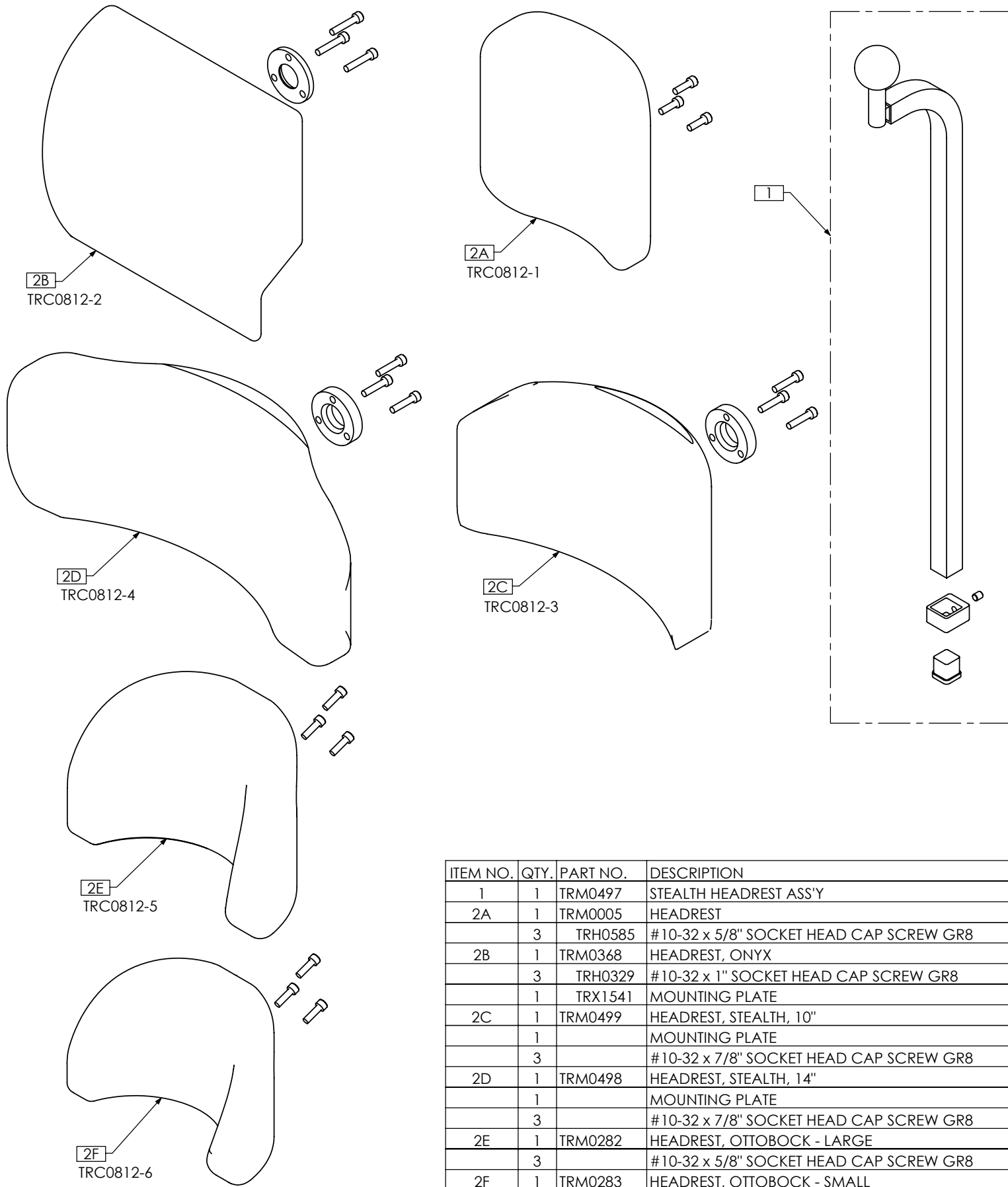


ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRM0497	STEALTH HEADREST ASS'Y
2A	1	TRM0005	HEADREST
	3	TRH0585	#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
2B	1	TRM0368	HEADREST, ONYX
	3	TRH0329	#10-32 x 1" SOCKET HEAD CAP SCREW GR8
	1	TRX1541	MOUNTING PLATE
2C	1	TRM0499	HEADREST, STEALTH, 10"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
2D	1	TRM0498	HEADREST, STEALTH, 14"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
2E	1	TRM0282	HEADREST, OTTOBOCK - LARGE
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
2F	1	TRM0283	HEADREST, OTTOBOCK - SMALL
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8

TRC0813 - HEADREST ASS'Y, BASIC


REVISION: 05/04/2007

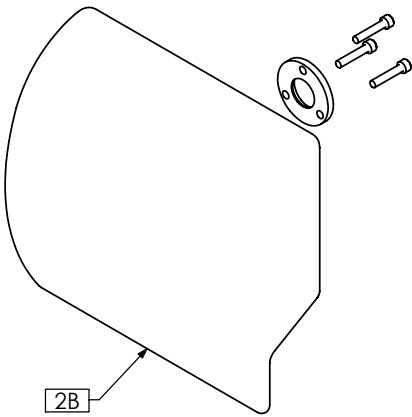


2B
TRC0813-2

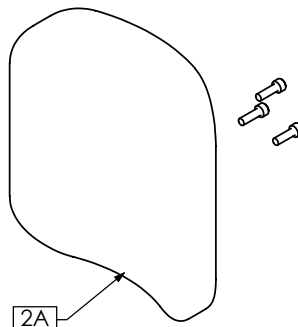


2A
TRC0813-1

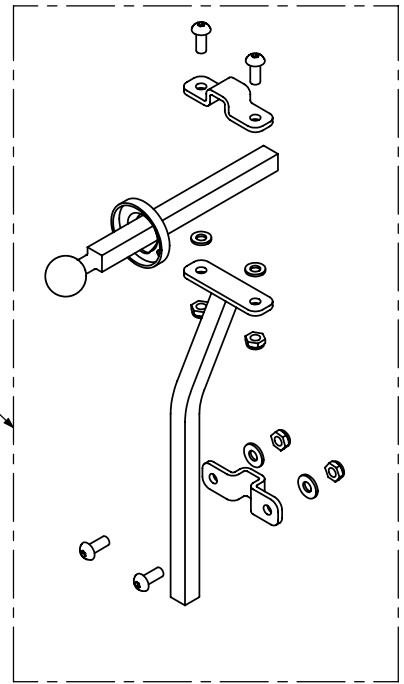


1

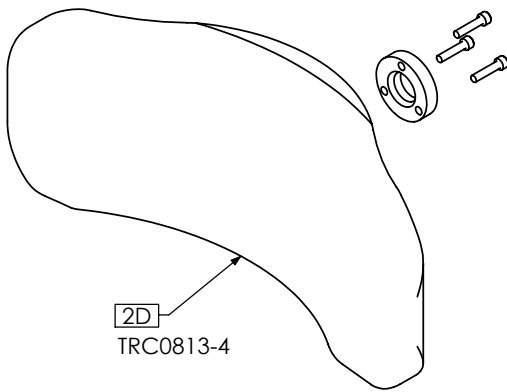


2D
TRC0813-4

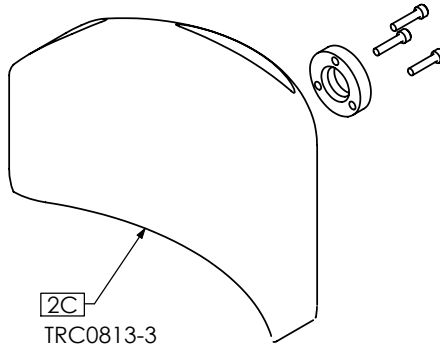

2C
TRC0813-3

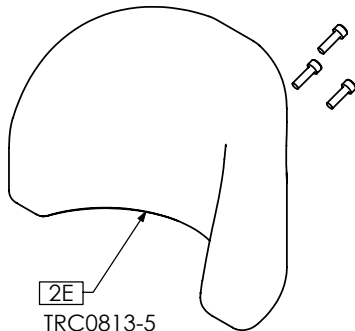

2E
TRC0813-5

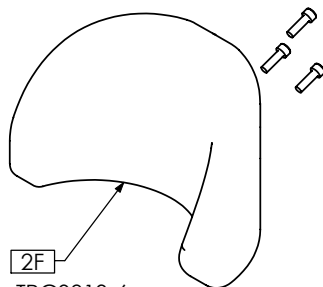


2F
TRC0813-6

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRC0435	HEADREST HARDWARE ASS'Y
	2	TRH0101	1/4" FLAT WASHER
	4	TRH0308	1/4-20 x 5/8" BUTTON SOCKET CAP SCREW
	4	TRH0202	1/4-20 SHALLOW NYLOC NUT
	2	TRH0102	1/4" NYLON WASHER
	1	TRM0040	HEADREST MOUNTING RING
	1	TRM0037	SMALL HORIZONTAL HEADREST ROD
	2	TRX2539	CLAMP
2A	1	TRM0005	HEADREST
	3	TRH0585	#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
2B	1	TRM0368	HEADREST, ONYX
	3	TRH0329	#10-32 x 1" SOCKET HEAD CAP SCREW GR8
	1	TRX1541	MOUNTING PLATE
2C	1	TRM0499	HEADREST, STEALTH, 10"
	1	TRX2540	MOUNT WLD
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
2D	1	TRM0498	HEADREST, STEALTH, 14"
	1		MOUNTING PLATE
	3		#10-32 x 7/8" SOCKET HEAD CAP SCREW GR8
2E	1	TRM0282	HEADREST, OTTOBOCK - LARGE
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8
2F	1	TRM0283	HEADREST, OTTOBOCK - SMALL
	3		#10-32 x 5/8" SOCKET HEAD CAP SCREW GR8

TRC0786 - HEADREST CLAMP ASS'Y


REVISION:05/11/2007

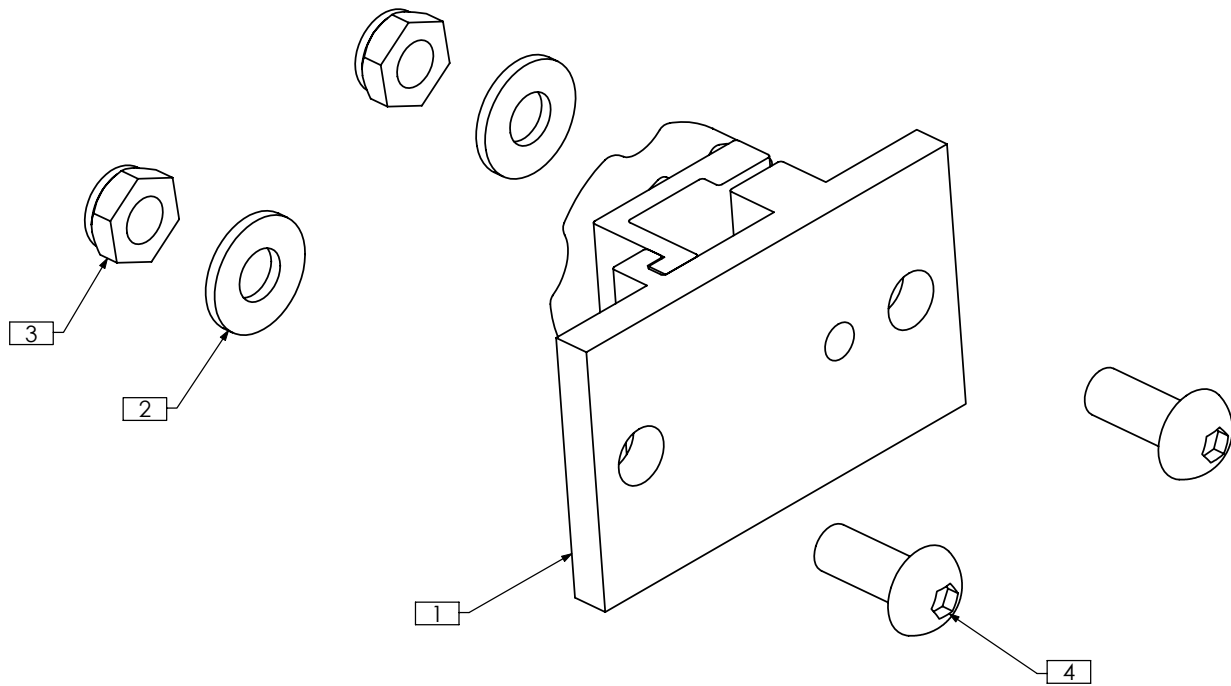

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRM0036	HEADREST CLAMP ASS'Y
2	2	TRH0101	1/4" FLAT WASHER
3	2	TRH0202	1/4-20 SHALLOW NYLOC NUT
4	2	TRH0308	1/4-20 x 5/8" BUTTON SOCKET CAP SCREW

TRC0787 - HEADREST CLAMP ASS'Y w/ SHIM


REVISION:05/24/2007

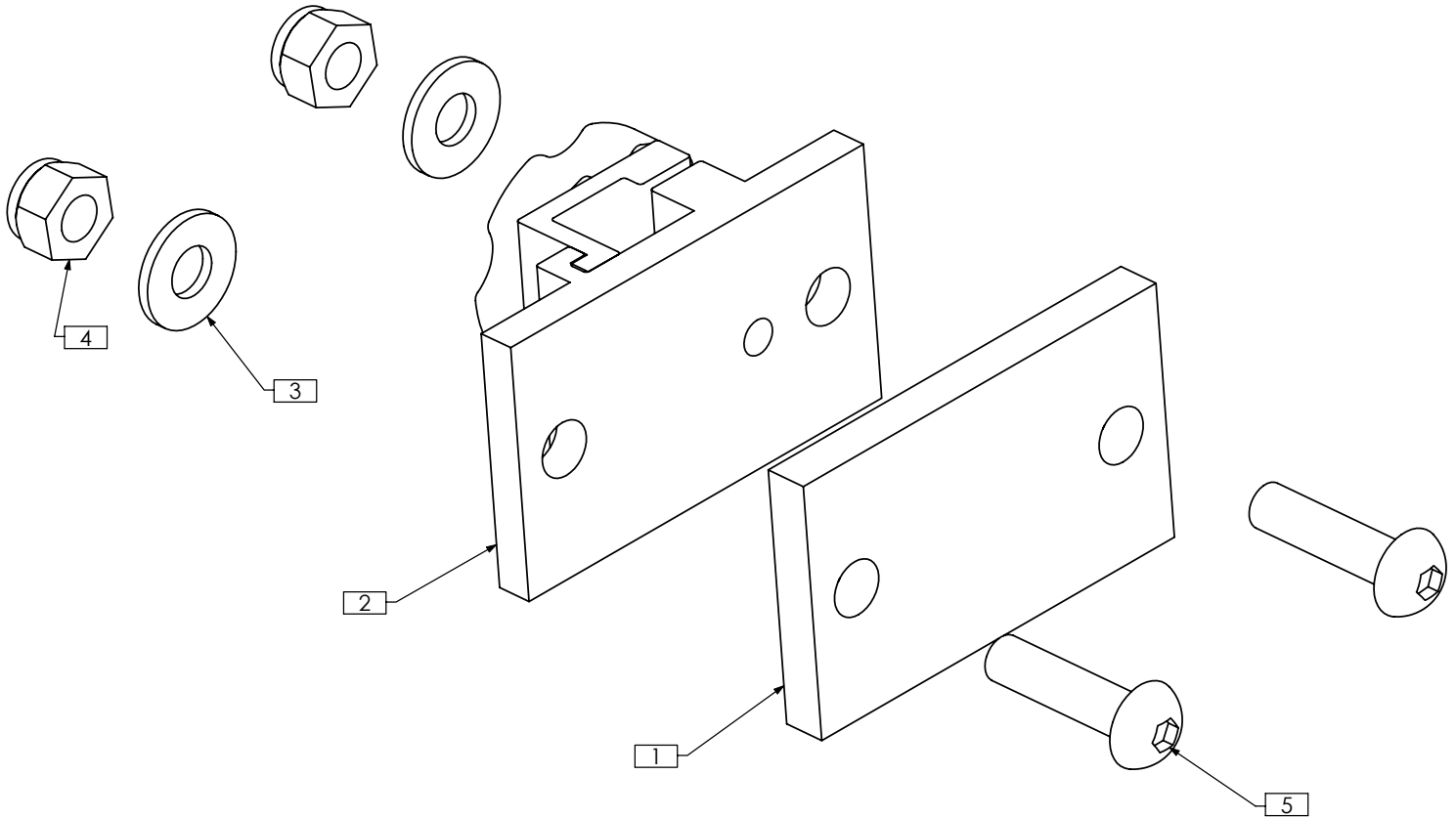

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX2685	SHIM
2	1	TRM0036	HEADREST CLAMP ASS'Y
3	2	TRH0101	1/4" FLAT WASHER
4	2	TRH0201	1/4-20 NYLOC NUT
5	2	TRH0450	1/4-20 x 1" BUTTON SOCKET CAP SCREW

TRC0788 - HEADREST CLAMP ASS'Y, TRIANGULAR MULTI-OFFSET


REVISION: 07/03/2007

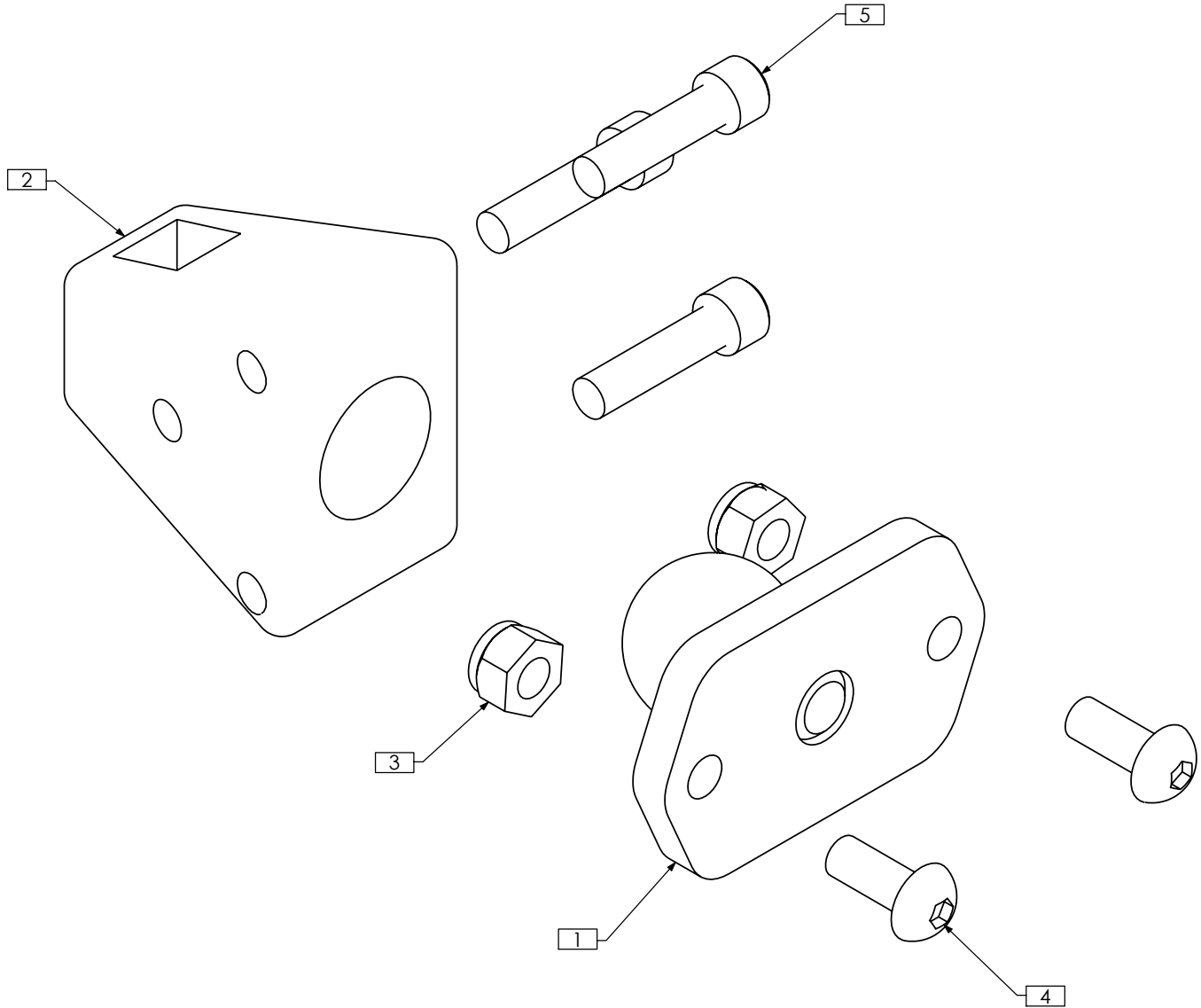

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX1617	TRX PIVOT BLOCK MNT WLD
2	1	TRM0039	TRIANGULAR MULTI-OFFSET
3	2	TRH0201	1/4-20 NYLOC NUT
4	2	TRH0308	1/4-20 x 5/8" BUTTON SOCKET CAP SCREW
5	3	TRH0335	1/4-20 x 1" SOCKET HEAD CAP SCREW GR8

TRC0398 - POWER HEADREST

REVISION: 09/14/2009

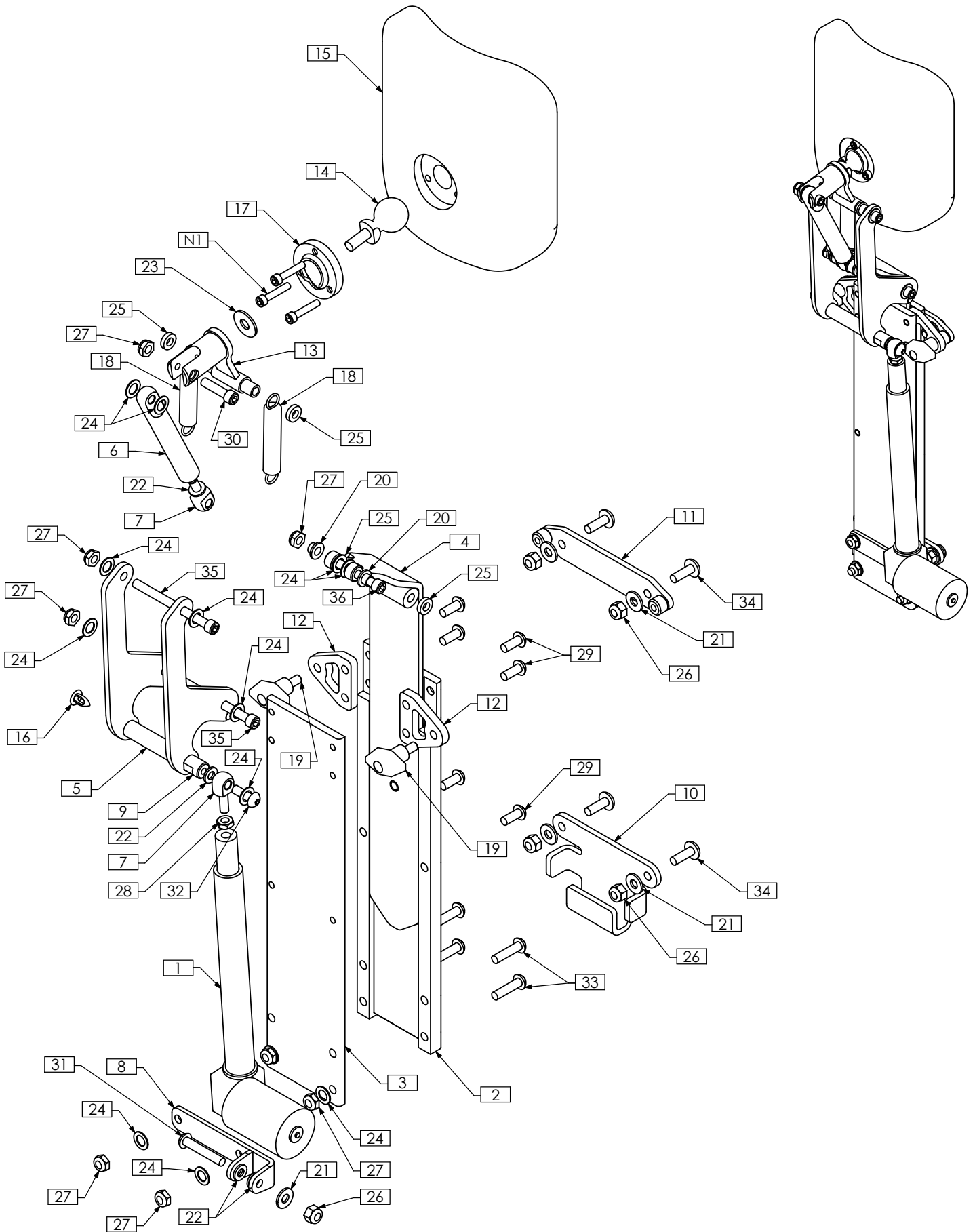

TRC0398 - POWER HEADREST

REVISION: 09/14/2009

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRA0235	6" STROKE POWER LEGREST ACTUATOR, RIGHT
2	1	TRX2110	FRONT CHANNEL
3	1	TRX2111	REAR PLATE
4	1	TRX2113	SLIDER PLATE WLD
5	1	TRX2118	PIVOT WLD
6	1	TRX2126	LINK ROD
7	2	TRX2127	ROD END
8	1	TRX2128	ACTUATOR MOUNT WLD
9	1	TRX2141	SPACER EXTENSION
10	1	TRX2300	LOWER MOUNT
11	1	TRX2301	UPPER MOUNT WLD
12	2	TRX2304	MOUNTING TAB
13	1	TRX2375	ADJUSTABLE HEADREST MOUNT WLD
14	1	TRX2376	HEADREST BALL
15	1	TRM0005	HEADREST
16	1	TRM0033	0.290 X 0.565 ARROW CLIP, BLACK NYLON
17	1	TRM0040	HEADREST MOUNTING RING
18	2	TRM0450	EXTENSION SPRING
19	2	TRM0770	KNOB, 1/4-20 X 1/2
20	2	TRH0008	MACHINE SCREW INSULATOR, 1/4"
21	5	TRH0101	1/4" FLAT WASHER
22	4	TRH0102	1/4" NYLON WASHER
23	1	TRH0109	5/16" NYLON WASHER
24	13	TRH0120	11/32" x 9/16"OD x 0.031" TH FLAT WASHER
25	4	TRH0133	1/4"ID x 1/2"OD x 0.125" NYLON WASHER
26	5	TRH0201	1/4-20 NYLOC NUT
27	8	TRH0202	1/4-20 SHALLOW NYLOC NUT
28	1	TRH0249	1/4-28 SHALLOW HEX JAM NUT
29	6	TRH0308	1/4-20 x 5/8" BUTTON SOCKET CAP SCREW
30	1	TRH0335	1/4-20 x 1" SOCKET HEAD CAP SCREW GR8
31	1	TRH0420	1/4-20 x 1 1/2" BUTTON SOCKET CAP SCREW
32	1	TRH0445	1/4-20 x 3/4" BUTTON SOCKET CAP SCREW
33	4	TRH0450	1/4-20 x 1" BUTTON SOCKET CAP SCREW
34	4	TRH0467	1/4-20 x 3/4" PHILLIPS TRUSS MACHINE SCREW
35	2	TRH0479	1/4-20 x 3" SOCKET HEAD CAP SCREW GR8
36	1	TRH0483	1/4-20 x 1 3/4" SOCKET HEAD CAP SCREW GR8

NOTES:

1. HARDWARE SUPPLIED WITH TRM0005