



POK

Pelvic Obliquity Kit Installation Instructions

U.S.: 700 Ensminger Rd., Suite 112, Tonawanda, NY 14150
TEL: 716-447-0050 FAX: 716-447-0030

Canada: 84 Citation Dr. Units 1-7, Concord, ON L4K 3C1 Canada
TEL: 905-695-0134 FAX: 905.695.0138




PELVIC OBLIQUITY KIT (POK) - INSTALLATION

The Pelvic Obliquity Kit includes two tapered 1/2" (13mm) foam pads that may be applied to the Matrx-Vi, Kidabra-Vi or Matrx-PS cushions:

POK Installation Instructions:

- 1) Remove the outer and inner covers.
- 2) Using the score lines, trim the obliquity pad(s) to the appropriate cushion width; 12" (31cm), 14" (36cm), 16" (41cm), 18" (46cm), 20" (51cm). **Note:** Trimming is not required for 22" (56 cm) cushion widths.
- 3) For 1/2" (13mm) obliquity build-up, use one pad only; For 1" (26mm) obliquity build-up, use two pads.

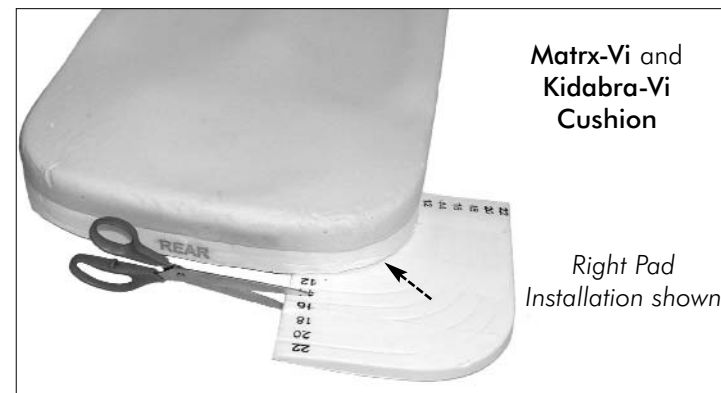

- 4) Remove paper backing from the adhesive strips (top surface). (For 1" (26mm) pelvic obliquity, adhere both pads together first with their tapered edges aligned with each other).
- 5) Align and adhere the obliquity pad(s) to the underside of the cushion foam on the right or left side of the cushion (as desired). **Note:** For left obliquity pad installation, the letter "L" (not shown) must be facing upward.
- 6) Re-install the inner and outer covers.

