Invacare® Matrix® FSSH

EN	User Manual	2
FR	Manuel d'utilisation	8
DE	Gebrauchsanweisung	14
IT	Manuale Utente	20
ES	Manual de usuario	26
PT	Manual de utilização	32
DA	Brugsanvisning	38
SE	Bruksanvisning	44
FI	Käyttöopas	50
CZ	Montážní návod	56
NL	Installatiehandleiding	62
EE	Paigaldusjuhend	68
LT	Montavimo vadovas	74

matrix[™]
SEATING SERIES

Yes, you can.®

TRD0238-E-05

EN

Table of Contents

- 1.0 Tools Required
- 2.0 Hardware Kit Contents
- 3.0 Pre-Installation Set-Up and Adjustment
 - 3.1 Mounting Clamp and Plate Adjustments
- 4.0 MaTRx Back Installation
 - 4.1 MaTRx PB (Posture Back) and MaTRx PB-Deep
 - 4.2 MaTRx PB-Heavy Duty and MaTRx PB-HD Deep
 - 4.3 MaTRx PB-Elite

1.0 Tools Required

Wrench (22mm): (provided)
Torx Key (T25): (provided)
Hex Key (2mm): (provided)
Utility Knife/Scissors
Phillips Screwdriver
Tape Measure/Ruler

2.0 Hardware Kit Contents

2.1 Cane Clamp Kit/Hardware


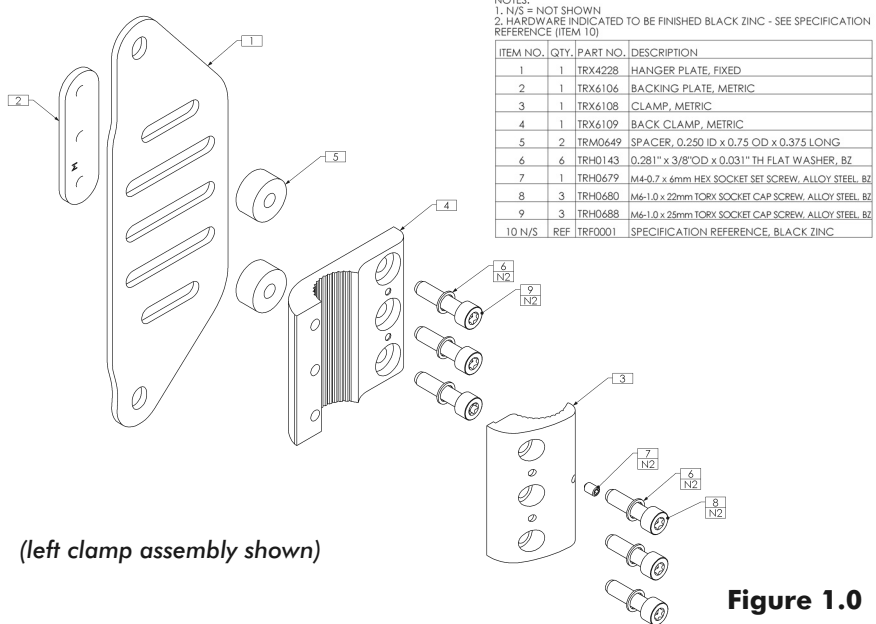
Figure 1.0

Figure 1.0 above provides an overview of the Space Saver Cane Clamp Kit for mounting our MaTRx back cushions onto the back canes of a wheelchair (*left clamp assembly is shown*). Each Hardware Kit includes *extra* mounting hardware (i.e.; screws/spacers/washers) to allow proper centering of the MaTRx back between the back canes and mounting plates.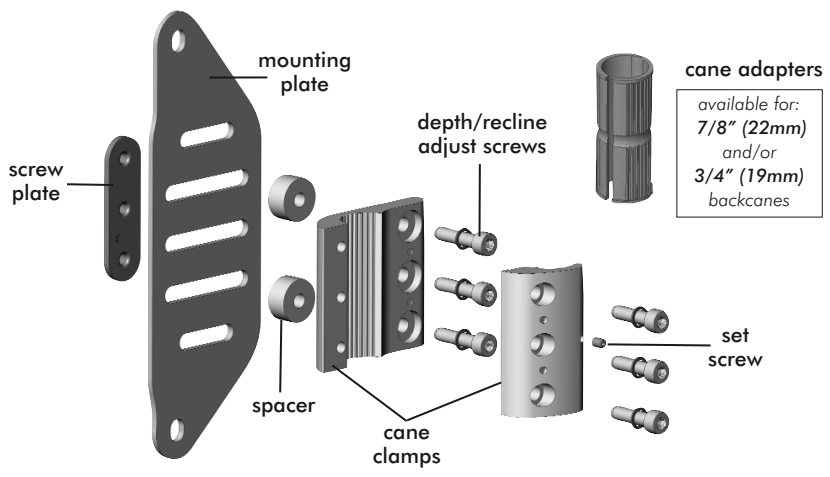

3.0 Pre-Installation Set-Up and Adjustment

3.1 Mounting Clamp and Plate Adjustments:

i) SEAT DEPTH ADJUSTMENT:

The **Cane Clamp Kits** may be mounted in a *forward* or *rearward* position on the back canes. The cane clamp kits are factory assembled in the *rearward* mounting configuration- see **Figure 3A** below. The slotted mounting plate design provides up to 1.5" (38mm) of fore and aft adjustment relative to the cane clamp.

The rearward cane clamp configuration allows for maximum seat depth on the wheelchair (up to 2.5" (64mm) maximum rearward offset, relative to the back canes).

By *inverting* the cane clamps on the back canes, the assembly can be reconfigured to provide a *forward* mounting position relative to the back canes* (up to a maximum 2.5" (64mm) forward offset)- see **Figure 3B** below.

**note: in the forward position, the mounting plates must also be removed and reinstalled in the proper orientation - as indicated below.*

! PLEASE NOTE

When establishing the mounting position for the MaTRx back, ensure the stability of the wheelchair is not compromised.

Adjusting the fore and aft position of the backrest will alter the user's center of gravity within the wheelchair. A recessed back position can significantly reduce the rearward stability of the wheelchair. Similarly, a more forward back position will reduce the wheelchair's forward stability.

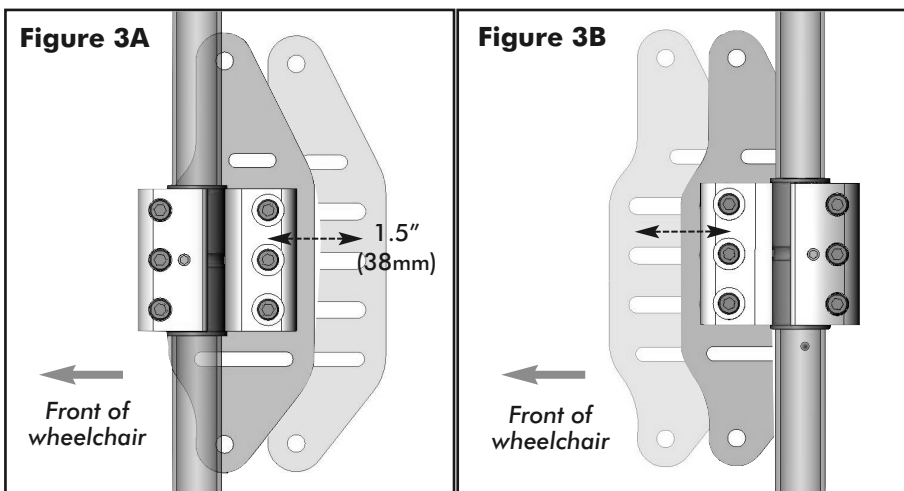
Figure 3A
Mounting Plate in the **rearmost** back position (maximized seat depth).

Figure 3B
Mounting Plate in the **forward most** back position.


3.0 Pre-Installation Set-Up and Adjustment

3.1 Mounting Clamp and Plate Adjustments ...cont'd:

ii) BACK HEIGHT ADJUSTMENT:


The height of the mounting clamp (Back Cushion) may be adjusted along the back canes by loosening the clamp hardware and sliding the complete assembly up or down to the desired height- refer to **Figure 3** at right.

(Note: to ensure proper set-up, the mounting plates/clamps must be installed at the same height on each back cane).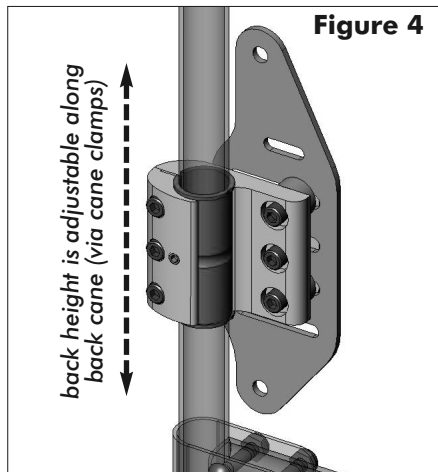

iii) BACK ANGLE ADJUSTMENT:

The slotted mounting plates are also designed to allow simple back angle adjustment for the MaTRx back. By loosening the mounting hardware, the plates can be rotated to alter the back angle and better conform to the needs of the client (see **Figure 4** at right).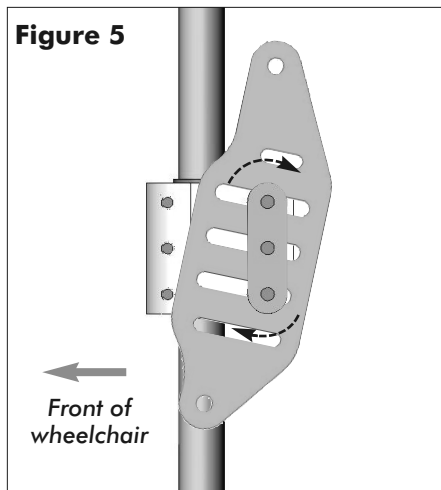

Back angle adjustment on a Cane Clamp assembly.

4.0 MaTRx Back Installation

ⓘ Important! Prior to installation, ensure the left and right mounting plates are secured to the back canes at the same height.


ⓘ Important! To function effectively and provide proper support, all MaTRx backs should be centered between the back canes.

The following section provides installation reference diagrams for the various MaTRx back styles. Using the space saver hardware kit provided, determine the combination of mounting hardware needed (*i.e.*; screws/spacers/washers) to properly center and install the MaTRx back onto the mounting plates.

note: in order to achieve the proper set-up, the mounting hardware required may vary slightly from the images shown.

4.1 MaTRx-PB and MaTRx-PB Deep

SPACE SAVER INSTALLATION* FOR THE MATRX-PB AND PB-DEEP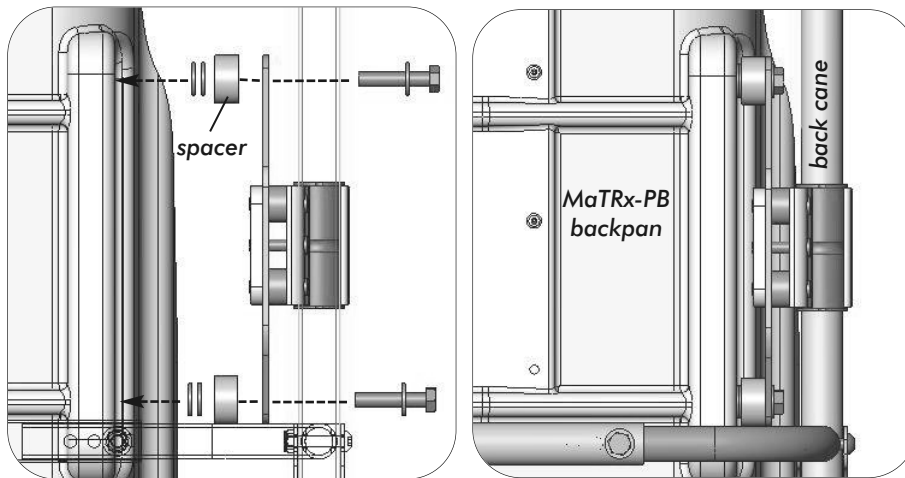


Please Note: the required mounting hardware may vary depending on the back size and back cane width- *images are provided for reference only.*

4.0 MaTRx Back Installation


4.2 MaTRx-PB Heavy Duty (HD) and PB HD-Deep

SPACE SAVER INSTALLATION* FOR THE MATRIX PB-HD AND HD-DEEP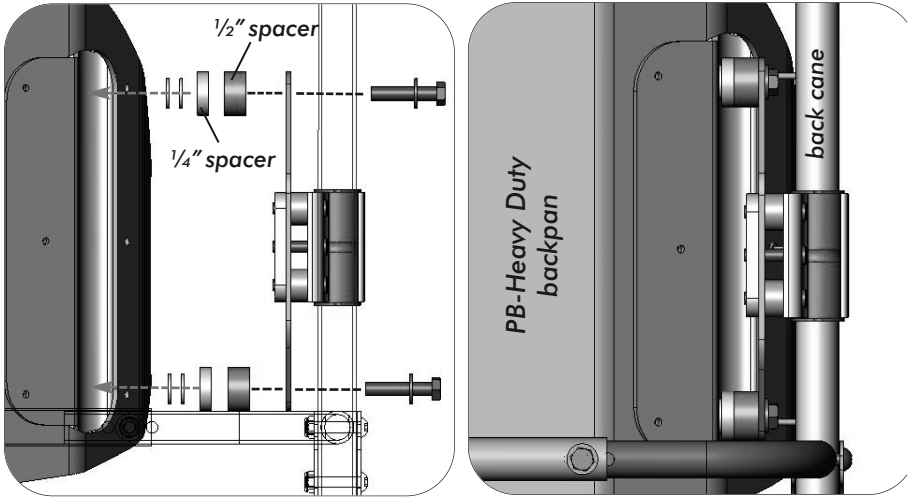

4.3 MaTRx-PB Elite

SPACE SAVER INSTALLATION* FOR THE MATRIX-PB ELITE

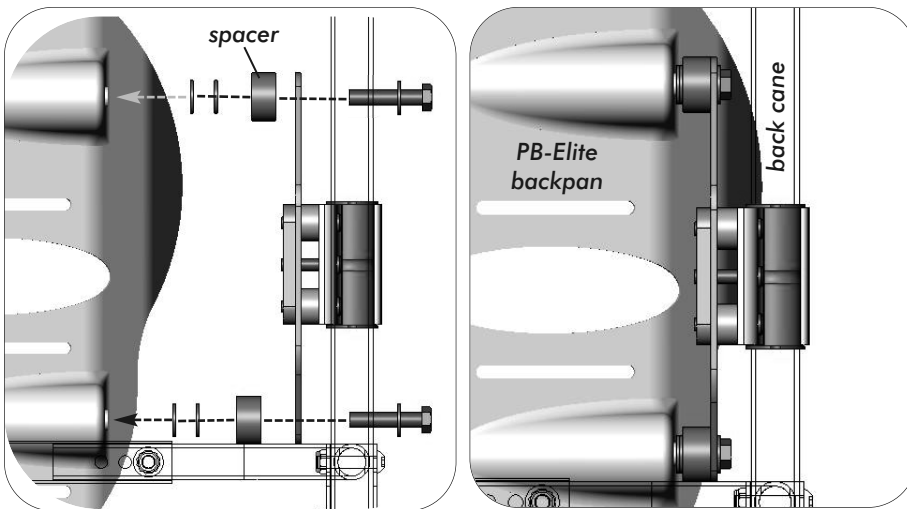***Please Note:** the required mounting hardware may vary depending on the back size and back cane width- images are provided for reference only.

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Canada • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4
Danmark • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00
Sverige & Suomi • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090
Österreich • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria