

INVACARE® MATRX®

PB
BACK



The **Invacare® Matrx® PB Back** is contoured to provide midline support and positioning. The extensive range of sizes provides exceptional support and comfort for a variety of individuals.

INVACARE® MATRIX® EASY SET HARDWARE

The **Invacare® Matrix® Easy Set hardware** is simple and easy to install, and all adjustments can be made with the individual seated in the wheelchair.




Features and Functional Benefits

- Quick release for easy removal/portability
- Infinite adjustment of depth (4"), angle (40°) and rotation
- Fits 3/4", 7/8" and 1" diameter back canes
- Hardware to fit 1 1/8" diameter canes also available


Styles: The following styles of the Invacare Matrix Backs are interchangeable on the Easy Set Hardware

1. Invacare® Matrix® Elite Back
2. Invacare® Matrix® Elite Deep Back
3. Invacare® Matrix® PB Back
4. Invacare® Matrix® PB Deep Back

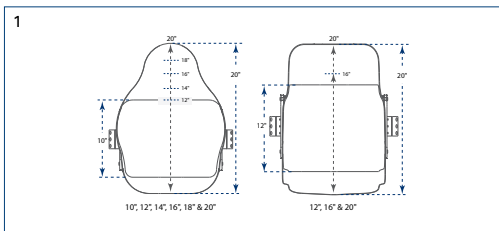


1. Heights

The Easy Set hardware accommodates a wide range of back heights to meet the support needs of the individual

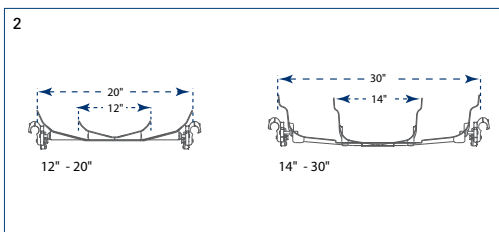

2. Widths

The Easy Set hardware fits back widths from 12"-30" to provide exact fit for each individual

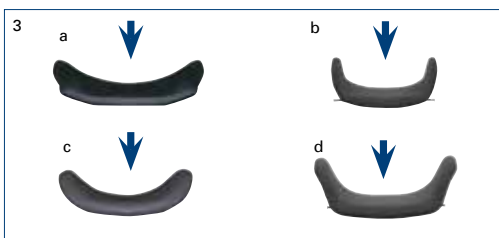

3. Contour Depth

The Easy Set hardware accommodates all four back shell styles with four choices of contour depths for appropriate midline support requirements

- a. Invacare® Matrix® Elite Back - 3" Contour Depth
- b. Invacare® Matrix® Elite Deep Back - 6" Contour Depth
- c. Invacare® Matrix® PB Back - 5" Contour Depth
- d. Invacare® Matrix® PB Deep Back - 7" Contour Depth




1. **Depth**
Adjusts 2" behind back canes
2. **Depth**
Adjusts 2" in front of back canes



3. **Angle**
40° range of anterior/posterior adjustment
4. **Angle**
40° range of anterior/posterior adjustment



- Rotation**
Back may be oriented to accommodate pelvic/trunk rotation

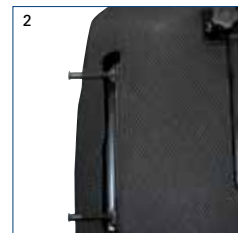

1. **Standard Mounting Pins**
Standard mounting pins included with all Invacare® Matrx® PB and Elite backs allow the Invacare® Matrx PB® back to fit chairs 1" narrower or same model width. Allows the Elite Back to fit chairs the same width

2. **Longer Mounting Pins**
Longer mounting pins included with all Invacare Matrx PB/Elite backs allow the Invacare Matrx PB and Elite Backs to fit chairs 1" wider



3. **Extended Mounting Pins**
Extended mounting pins available for Invacare Matrx PB & Elite Backs Allow the back to fit chairs up to 2" wider

(Not available on backs with greater than 300 lb weight capacity)

4. **Extended Range Mounting Hardware Upgrade**
Provides 1.75" of additional depth adjustment compared to the standard mounting hardware. Available on all models of Posture Backs and Elite Backs – Easy Set, Heavy-Duty or Fixed Space Saver hardware



5. **Heavy Duty Easy Set Mounting Hardware**
Standard with Heavy-Duty Posture Backs. Recommended for users over 300 lb., high tone or heavy use

6. **Fixed Space Saver Mounting Hardware**
Requires tools for back removal. Saves space for tighter width applications. Fully adjustable. Recommended for power wheelchair application. Available as a no charge option



7. **Split Mounting Hardware**
Provides clearance of mid back cane rigidizer bars. Also allows Invacare Matrx Backs to mount without interference of cantilever armrests. Available as a no charge option. Not recommended for Invacare Matrx Backs wider than 20". (Not available on backs with greater than 300 lb. weight capacity)

8. **Extended Shear Reduction Mounting Hardware**
Allows all Invacare Matrx backs to be mounted directly to Motion Concepts TRx seating with Power Recline and ESR