Invacare® Matrx® MX2 Back

en	Backrest	
	User Manual	4
de	Rückenlehne	
	Gebrauchsanweisung	10
fr	Dossier	
	Manuel d'utilisation	17
it	Schienale	
	Manuale d'uso	24
es	Respaldo	
	Manual del usuario	31
pt	Encosto	
	Manual de utilização	38
nl	Rugleuning	
	Gebruiksaanwijzing	45
da	Ryglæn	
	Brugsanvisning	52
no	Rygglene	
	Bruksanvisning	58
sv	Ryggstöd	
	Bruksanvisning	64
fi	Selkänojan	
	Käyttöopas	71

en This manual may not be reproduced or reprinted either partly or completely without previous written consent from Motion Concepts or its statutory representatives.

This user manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make any necessary as they become necessary. Any changes made to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. If you have any questions or concerns regarding the information provided in this manual, please contact our Customer Service Department for assistance.

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1.1 OVERVIEW

SAFETY SYMBOLS:

Proper set-up and safe use of your Matrx® MX2 Back depends on your own good judgement and common sense, as well as that of your provider, caregiver and/or health professional. The following symbols are used throughout this user manual to identify warnings, cautions and important notes. It is very important that you read and understand them completely.

 **CAUTION!** Failure to heed the cautions in this manual may result in damage to your Matrx Seating product.

 **WARNING!** Failure to heed the warnings in this manual may result in death, serious injury or damage.

 **IMPORTANT!** Important information to remember during installation, set-up and/or adjustment of you Matrx Seating product.

USER WARNING!

Improper use of this product may cause injury or damage. DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a health care professional or service provider before attempting to use this equipment- otherwise damage or injury may occur.

In the unlikely event that a serious incident occurs in relation to this device, the incident should be reported to Motion Concepts and your Local Service Provider as soon as possible.

IMPORTANT INFORMATION

The best way to avoid problems related to pressure injuries is to understand their causes and your role in a skin management program. Your therapist and physician should be consulted if you have questions regarding individual limitations and needs. All back supports should be selected carefully. Working with your therapist and physician is the best way to assure that a cushion choice matches your individual needs. As the needs of the individual become more complex, the evaluation becomes more important.

 **IMPORTANT!** Please refer to the Wheelchair Owners/Operators Manual for additional safety information, warnings and cautions related to the operation of your wheelchair.

1.2 INTENDED USE

The Matrx® MX2 back is intended to provide precise orientation within a wheelchair for optimal postural support for active users. The carbon fiber back shell is specially engineered to provide unmatched stability for peak performance, and the foam is oversized to provide extra comfort and protection.

The Matrx® MX2 back includes a movable foam lumbar pad that can be installed/inserted behind the existing foam cushion to provide additional positioning capability. Refer to **Section 3.3- Lumbar Support**.

2.0 SAFETY INFORMATION

2.1 TRANSPORTATION WARNINGS!

⚠ WARNING! Risk of death or severe injury when transporting a client in a wheelchair fitted with a Matrx® MX2 back in a motor vehicle.

- The Matrx MX2 Back is not designed for use with a headrest support and is NOT APPROVED for Occupied Transport in a motor vehicle.
- Motion Concepts recommends that the client be transferred out of the wheelchair and secured into a vehicle seat using the vehicle lap and diagonal seat belt.

⚠ WARNING!

Risk of serious injury or damage when transporting a wheelchair fitted with a Matrx® MX2 back in a motor vehicle.

- Matrx® MX2 backs may only be transported in a vehicle when used in conjunction with wheelchairs that comply with the performance requirements of RESNA WC19 or ISO 7176-19.
- For safe transportation of a wheelchair in a motor vehicle, refer to the user manuals of the wheelchair manufacturer and wheelchair tie-down system.

i IMPORTANT! Prior to transport, ensure that the Matrx MX2 Back is securely fastened into the wheelchair in accordance with Motion Concepts Installation Instructions.

i IMPORTANT! If the wheelchair and Matrx back is transported unoccupied, the wheelchair must be secured in a forward facing position, using approved tie down straps, in accordance with the wheelchair and tie-down manufacturers instructions. If the any equipment/components are dismantled for transportation, ensure that all parts are secured safely within the vehicle.

2.2 GENERAL SAFETY WARNINGS!

⚠ WARNING! Risk of Death, Serious Injury or Damage

Improper use of this product may cause injury or damage. DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment.

⚠ WARNING! Risk of Death, Serious Injury, or Damage

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury or damage. Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

- DO NOT smoke while using this wheelchair

⚠ WARNING! Risk of Serious Injury

The mounting position of a Matrx® back may alter the users center of gravity in the wheelchair. Ensure the stability of the wheelchair is not compromised while seated in the wheelchair. If you have any concerns with the stability of your wheelchair, discontinue use, and contact your healthcare professional or service provider for assistance.

⚠ WARNING! Risk of Serious Injury or Damage

- The hardware provided is high strength and tested for durability. **Do Not substitute hardware.** Use only the original hardware supplied with your Matrx back
- After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

⚠ WARNING! Risk of Injury

Matrx® products are specifically designed and manufactured for use in conjunction with Matrx® accessories. Accessories designed by other manufacturers have not been tested by Motion Concepts and are not recommended.

⚠ WARNING Risk of Injury

Skin condition should be checked very frequently after the installation of any new seating system. As the needs of the individual become more complex, the evaluation becomes more important. Working with you therapist, physician and service provider is the best way to assure that a seating choice meets your individual needs.

⚠ WARNING! Risk of Serious Injury or Damage

Following ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

3.0 MATRX MX2 BACK INFORMATION

3.1 MX2 BACK SPECIFICATIONS

CONFIGURED MODEL	DESCRIPTION	CONTOUR DEPTH	WEIGHT CAPACITY
IMX2wwhh	Matrx MX2 Back	3 in. (76mm)	250 lbs/ (113 kg)

* ww hh = width/height

Available Sizes: Width: 14" - 18" (355mm - 460mm) ;
Height: 6" (150mm), 9" (230mm), 12" (310mm), 16" (410mm)

For further information on any of our Matrx backs, including all available models/sizes, please visit www.motionconcepts.com or www.invacaredocs.com.

3.2 MX2 BACK INSTALLATION & SET-UP

i IMPORTANT! Installation and set-up of the Matrx MX2 Back **MUST** be performed by a Qualified Service Technician and/or Healthcare Professional.

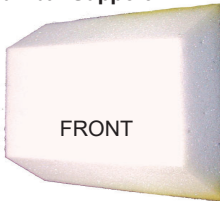
- The Matrx® MX2 Back has been designed to install onto back canes that are up to 1" (25mm) wider or 1" (25mm) narrower than the nominal back width.
- The Matrx® MX2 Back may be configured with Fixed Hardware or Removable hardware (optional)
- Installation of the Matrx® MX2 Back is limited to wheelchair back canes with an outside diameter greater than 1" (25mm) or less than 3/4" (19mm).

3.3 MX2 LUMBAR SUPPORT (Optional)

The optional **Lumbar Support** is a pre-fabricated foam insert that provides lower back support for additional comfort. The position of the lumbar insert can be adjusted inside the Matrx® MX2 cover (see *instructions below*), or it may be removed if no lumbar support is desired.

1. To access the lumbar support, open the hook and loop fastening strips on the outer cover at the bottom rear of the back shell.
2. The foam lumbar support is installed inside the Matrx® MX2 cover between the foam back cushion and the back shell .
3. The lumbar support can be adjusted to any desired height/position.
4. Refasten the cover (via hook and loop strips) onto the back pan.

Lumbar Support

***NOTE:** The lumbar foam may be cut/modified as needed, to meet the needs of the end user.

3.4 PRIVACY FLAP (Sold Separately)

Privacy Flaps are available (sold separately) for use with all standard sized MX2 backs. All standard sized backs come with a pre-sewn loop strip across the bottom rear of the cushion cover for installation of the privacy flap.

4.0 CARE AND MAINTENANCE

4.1 INSPECTION AND MAINTENANCE

Frequent visual inspection should be performed on all components, including upholstery, foam, hardware/fasteners and any plastic components.

WARNING! Risk of injury or damage

- Do not continue to use this product if problems or damage are discovered;
- Corrective maintenance can be performed by or arranged thru your local service provider

Weekly: Inspect **ALL** fasteners weekly to ensure that mechanical connections and attaching hardware are tightened securely.

Monthly: Perform a visual inspection of parts including hardware, brackets, upholstery materials, foam, and plastics for deformation, corrosion, breakage, wear or compression. Replace damaged/worn components if necessary.

4.2 CLEANING/LAUNDERING:

The outer cover on the MX2 Back is made of a breathable spacer mesh, combined with a leatherette material for added durability. Regular cleaning and inspection of the outer cover is recommended.

WARNING! Risk of damage

To avoid infectious exposure, inspect cushion thoroughly and properly clean and disinfect any materials soiled with blood or other bodily fluids.

CAUTION! Risk of damage

- DO NOT use cleaning and disinfecting agents with abrasive, staining, or polymer-damaging properties such as phenols, alcohols, or bleaches. Chlorine solutions used even in low concentrations on a regular basis can diminish the life of the cover.
- Chosen cleaning agents and disinfectants must be compatible with one another in order to effectively protect the materials on which they are applied.
- Keep clear of open heat sources during cleaning.

4.2.1 GENERAL CLEANING:

OUTER COVER: The MX2 cover can be wiped with a damp absorbent towel, cloth, or paper towel. Remove all gross contamination from surface (i.e. food particles, urine, feces, blood). Spot-clean with mild soap and water.

BACK SHELL/HARDWARE: The back shell and hardware/components can be wiped clean with soap and water as necessary.

4.2.2 LAUNDERING INSTRUCTIONS

 **IMPORTANT!** PRIOR to Laundering: The MX2 Back cover **MUST** be removed from the inner foam cushion. (**Note:** Do Not Dry Clean, Do Not Bleach).

 **IMPORTANT!** Adequate drying is a crucial factor in the care and maintenance of all covers. The covers should be dried completely prior to use on the cushion. Air drying of covers is recommended.

I. MACHINE LAUNDERING OR HAND-WASHING INSTRUCTIONS (OUTER COVER)

- Carefully remove the MX2 Back cover from the back shell by releasing the velcro hook & loop fasteners along the top, bottom and inside of the back shell.
- Carefully remove the inner foam cushion from the inside of the MX2 Back cover.
- Outer covers can be hand-washed or machine-washed in warm water not higher than 60°C (140°F), on a delicate cycle with neutral detergents such as Tide or Persil.
- To avoid possible damage, ensure the cover does not come in contact with any other sharp garment accessories during laundering.

II. DRYING INSTRUCTIONS (OUTER COVER)

Adequate drying is a crucial factor in the care and maintenance of the outer cover. MX2 covers should be dried completely prior to use. Air drying is the method of choice.

- DO NOT WRING OUT.
- Hang cover and allow it to drip dry to minimize wrinkling.
- Alternatively, covers may be tumble-dried on low/cool setting.
- DO NOT IRON OR PRESS covers

4.3 REUSE/RECYCLING

WARNING! Risk of Injury

Always have an authorized dealer inspect the product for damage or wear PRIOR to transferring the product to a different user. If any damage is found, DO NOT use this product. Failure to heed these warnings may result in severe injury.

This product is suitable for re-use. The outer cover (fabric and foam), back shell and mounting hardware must be inspected by a qualified dealer or healthcare professional before re-use. If any damage is found, DO NOT RE-USE PRIOR to reuse, clean and disinfect the product thoroughly in accordance with local government regulations.

4.4 DISPOSAL

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations. Ensure that the back is cleaned prior to disposal to avoid any risk of contamination.

5.0 WARRANTY INFORMATION

WARRANTY

A manufacturer's warranty for our Matrx products is provided in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the product was purchased.

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Invacare Limited
Advena Ltd.