## Invacare® Matrix® ETAC Back Cane Mounting Hardware

<b>EN</b>	User Manual .....	2
<b>FR</b>	Manuel d'utilisation .....	8
<b>DE</b>	Gebrauchsanweisung .....	14
<b>ES</b>	Manual de usuario .....	20
<b>PT</b>	Manual de utilização .....	26
<b>IT</b>	Manuale Utente .....	32
<b>DA</b>	Brugsanvisning .....	38
<b>SE</b>	Bruksanvisning .....	44
<b>FI</b>	Käyttöopas .....	50

## Table of Contents

**EN**

- 1.0 Tools Required
- 2.0 Installation Overview
- 3.0 Before Installation - Important!
- 4.0 Mounting Hardware & Back Installation
- 5.0 ETAC Wheelchair Recline Adjustments (*for greater seat depth*)

### 1.0 Tools Required

Hex Key (5mm): (provided)

Hex Key (2mm): (provided)

Wrench (13mm)

Tape Measure/Ruler

### 2.0 - Installation Overview

Our **ETAC Mounting Hardware** has been uniquely designed to allow the mounting of our **MaTRx-PB** and **Matrix-PB Elite** Back Cushions\* onto an ETAC wheelchair. The ETAC hardware utilizes a unique mounting clamp, specially designed to install onto the back canes of an ETAC wheelchair. The remaining hardware utilizes the same components as our standard easy-set mounting hardware, thereby providing clients with the same ease of installation & adjustment they have grown accustomed to.

**i\*Important!** *the ETAC Mounting Hardware is **not recommended** for use with our Heavy-Duty Matrix backs.*

## 2.0 - Installation Overview


**i Important!** This manual provides installation and set-up instructions specific to the ETAC mounting hardware; Please also be certain to read through the User Manual provided with your Matrx back cushion for additional safety, installation and set-up instructions.

**! WARNING!** All hardware used by Motion Concepts in the manufacturing and installation of our products is high strength; Use only the hardware provided, **do not** substitute hardware.

### ETAC Mounting Hardware- Assembly Drawing

Figure 1.0

**(Note: Right Mounting Assembly Shown)**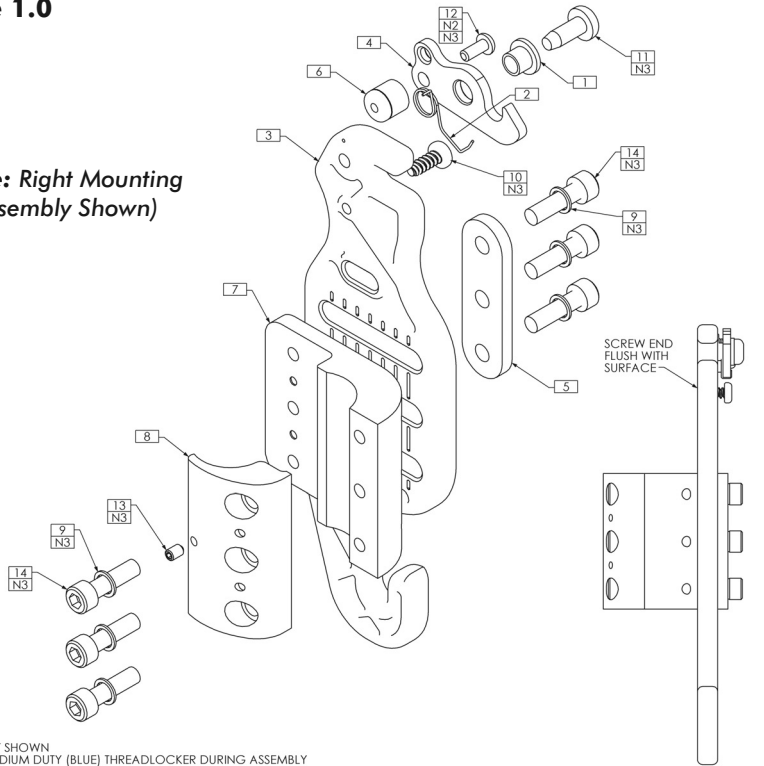


- NOTES:  
 1. N/S = NOT SHOWN  
 2. APPLY MEDIUM DUTY (BLUE) THREADLOCKER DURING ASSEMBLY  
 3. HARDWARE INDICATED TO BE FINISHED BLACK ZINC - SEE SPECIFICATION REFERENCE (ITEM 15)

ITEM NO.	QTY.	PART NO	DESCRIPTION	ITEM NO.	QTY.	PART NO	DESCRIPTION
1	1	TRX1893	LATCH SPRING SLEEVE	9	6	TRH0143	0.281" x 3/8"OD x 0.031" TH FLAT WASHER, BZ
2	1	TRX1894	LATCH SPRING, RIGHT	10	1	TRH0548	# 10 x 1/2" PAN TORX PLASTITE SCREW, BZ
3	1	TRX1898	BACK FRAME MOUNTING PLATE	11	1	TRH0549	M6 x 16 CHEESE POSIDRIVE TAP TITE SCREW, BZ
4	1	TRX5182	LATCH	12	1	TRH0678	M4-0.7 x 10mm BUTTON TORX CAP SCREW, ALLOY STEEL, BZ
5	1	TRX5966	SPACER	13	1	TRH0679	M4-0.7 x 6mm HEX SOCKET SET SCREW, ALLOY STEEL, BZ
6	1	TRX6107	LATCH STOP, METRIC	14	6	TRH0723	M6-1.0 X 22mm SOCKET HEAD CAP, ALLOY STEEL, BZ
7	1	TRX6949	BACK CLAMP, METRIC, ETAC	15	N/S	REF	TRF0001 SPECIFICATION REFERENCE, BLACK ZINC
8	1	TRX6950	CLAMP, METRIC, ETAC				


### 3.0 Before Installation- Important!

#### Exchanging the Left & Right Push Handle Assemblies:

In order to provide sufficient clearance for the Matrix Back mounting hardware, the left & right push handle assemblies must be exchanged so that the clamp assembly is re-positioned to the **outside** of the back canes. Refer to steps **1.** and **2.** below.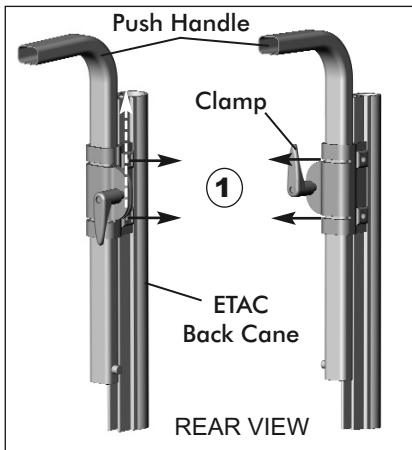

**1.** Remove the push handles from the ETAC back canes by loosening the 5mm bolts (x2) and sliding the push handle assembly up and off of the back cane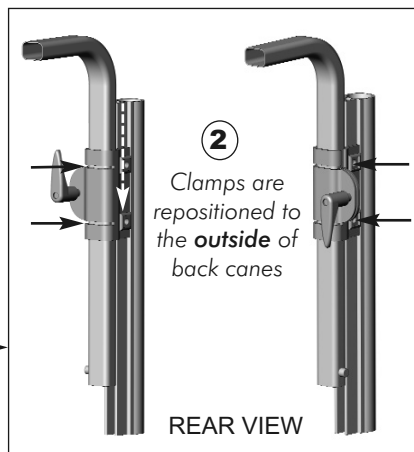

**2.** Reinstall the push handles onto the **opposite** back canes (so that the clamp is on the outside of the back cane). Refasten the push handle assembly via the 5mm bolts (x2).

### 4.0 Hardware and MaTRx Back Installation

**i Important!** Prior to installation, ensure the Push Handle Assemblies have been swapped per the instructions in **Section 3.0.**


**i Important!** To function effectively and provide proper support, all MaTRx backs should be centered between the back canes.

#### Installation Instructions:

- 1 Remove existing wheelchair back/upholstery (if applicable).
- 2 Ensure the mounting plates and mounting clamp assemblies are in the proper orientation (refer to **Figure 2.0**)



## 4.0 Hardware and MaTRx Back Installation

### Installation Instructions ... cont'd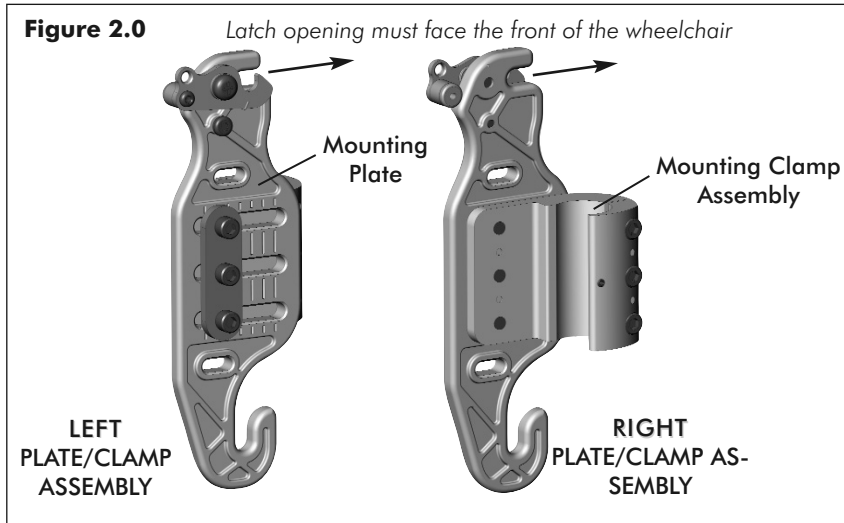

- 3 Working on one side (back cane) at a time, loosen the screws (x3) on the **ETAC Mounting Clamp** ass'y and slide the clamp assembly (*incl. mounting plate*) onto the back cane. Slide the mounting clamp ass'y to the desired height and tighten the 3 screws just enough to hold it in place. See **Figure 3A/3B** below.

*(Note: Mounting clamps can be fully tightened after the back is installed and all final adjustments have been made.)*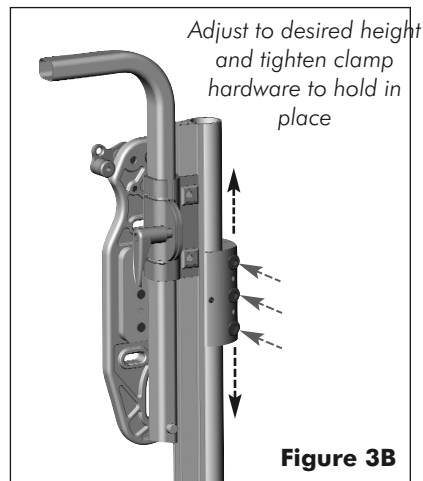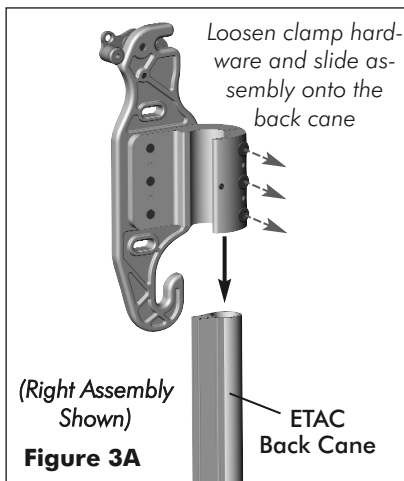

## 4.0 Hardware and MaTRx Back Installation

### Installation Instructions ... cont'd


- 4 Repeat step 3 for the opposite mounting clamp\*.

**i \*Important!** To ensure proper back installation, both mounting clamps must be installed at the **same height** on the back canes with the mounting plates **parallel** with each other.

- 5 To install your Matrix-PB or Matrix PB-Elite back onto the mounting plates:

**Step 1:** Slide the lower mounting pins of the back into the hooked slots at the bottom of each mounting plate (see **Figure 4.0**).

**Step 2:** With the lower pins in place, lock/snap the upper mounting pins into the latch mechanism at the top of the mounting plates.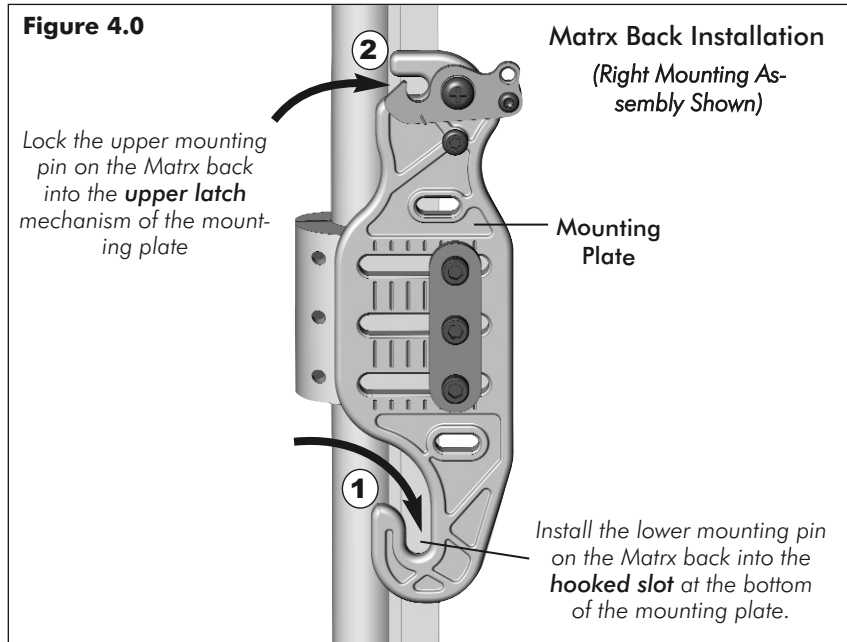

- 6 With the back cushion in place make any final adjustments to the back height (if necessary) via clamp hardware, ensuring that both clamps are adjusted equally.
- 7 With the back height set, fully tighten all clamp assembly hardware to secure the back into position.


**i Important!** The mounting pins for the MaTRx back may require adjustment to ensure the backrest is properly fitted/secured to the wheelchair; For instructions on adjusting the mounting pins, please refer to the installation/user manual provided with your MaTRx Series back.

## 5.0 ETAC Recline Adjustments (for greater seat depth)


**NOTE:** For users that require maximum depth for their back cushion, it may be necessary to adjust the recline angle of the ETAC back canes in order to maximize the seat depth, and provide better access to the adjustment hardware.

### Adjusting the Recline Angle of the Back Canes

- Image (1) illustrates a back installed at the maximum depth, and the ETAC back canes in their upright (non-reclined) position. (**Note:** As shown in this configuration, access to the adjustment hardware can be difficult)
- To increase the recline angle of the ETAC back canes, loosen the 13mm nut at the bottom of each back cane (2). Adjust the recline angle via the slotted channel at the bottom of each back canes (3). (**Note:** Ensure that both back canes are adjusted equally). Following the recline adjustments, the mounting plates may also require adjustment to compensate for revised back cane angle.

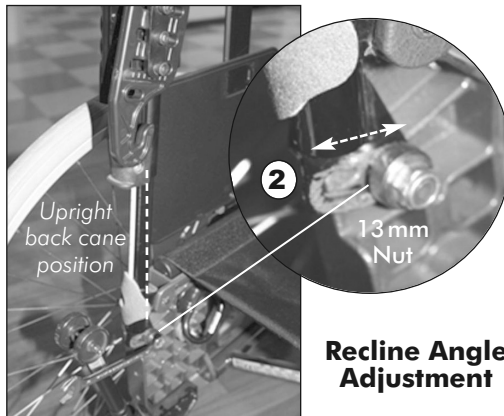**Recline Angle Adjustment**

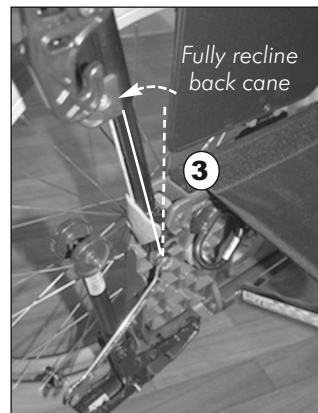- Image (4) shows the ETAC back canes in the reclined position with the ETAC mounting hardware re-adjusted to compensate for the new recline angle. This reclined back cane configuration creates more available space for depth adjustment, and improves access to the adjustment screws/hardware.

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
Figure 1.0

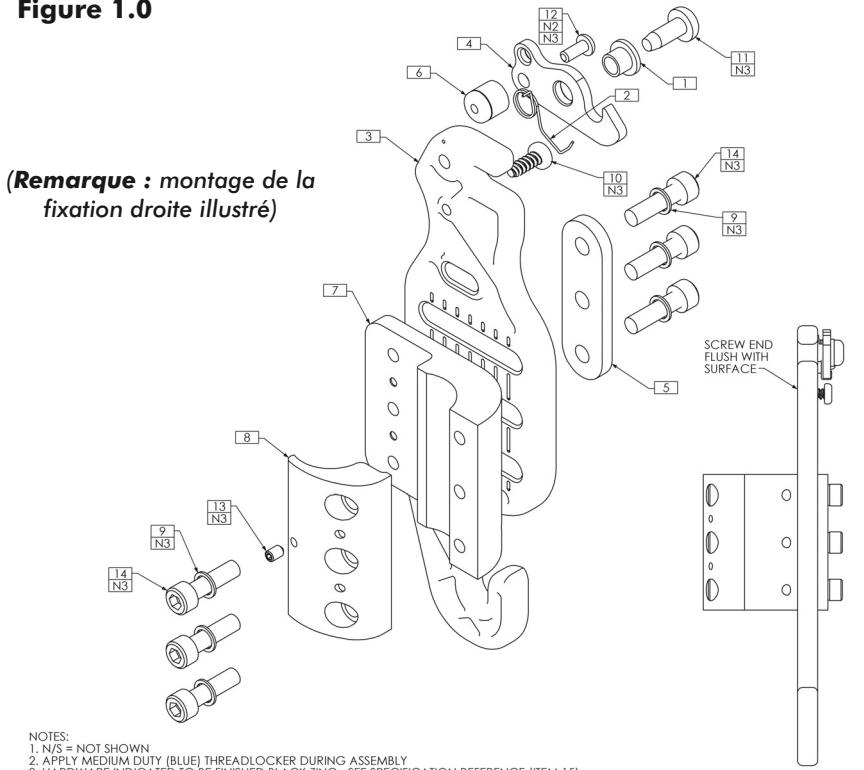NOTES:

1. N/S = NOT SHOWN
2. APPLY MEDIUM DUTY (BLUE) THREADLOCKER DURING ASSEMBLY
3. HARDWARE INDICATED TO BE FINISHED BLACK ZINC - SEE SPECIFICATION REFERENCE (ITEM 15)

ITEM NO.	QTY.	PART NO	DESCRIPTION	ITEM NO.	QTY.	PART NO	DESCRIPTION	
1	1	TRX1893	LATCH SPRING SLEEVE	9	6	TRH0143	0.281" x 3/8"OD x 0.031" TH FLAT WASHER, BZ	
2	1	TRX1894	LATCH SPRING, RIGHT	10	1	TRH0548	#10 x 1/2" PAN TORX PLASTITE SCREW, BZ	
3	1	TRX1898	BACK FRAME MOUNTING PLATE	11	1	TRH0549	M6 x 16 CHEESE POSIDRIVE TAP TITE SCREW, BZ	
4	1	TRX5182	LATCH	12	1	TRH0678	M4-0.7 x 10mm BUTTON TORX CAP SCREW, ALLOY STEEL, BZ	
5	1	TRX5966	SPACER	13	1	TRH0679	M4-0.7 x 6mm HEX SOCKET SET SCREW, ALLOY STEEL, BZ	
6	1	TRX6107	LATCH STOP, METRIC	14	6	TRH0723	M6-1.0 X 22mm SOCKET HEAD CAP, ALLOY STEEL, BZ	
7	1	TRX6949	BACK CLAMP, METRIC, ETAC	15	N/S	REF	TRF0001	SPECIFICATION REFERENCE, BLACK ZINC
8	1	TRX6950	CLAMP, METRIC, ETAC					

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**Canada** • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4
**Danmark** • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00
**Sverige & Suomi** • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090
**Österreich** • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria