


# PRIVACY FLAP (PFLAP) INSTALLATION INSTRUCTIONS

**Optional Privacy Flaps** are available for use with all standard sized Matrix back supports. Each Matrix back comes with a loop strip sewn across the bottom rear of the cover for installation of the privacy flap. The opposite end of the PFLAP (hook strip) is secured to the wheelchair seat. (See size chart below).

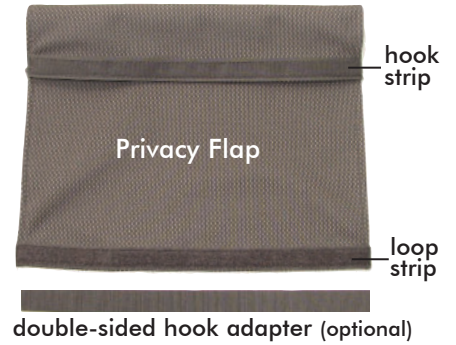


Privacy Flap Sizing (Width x Length)					
PFLAP-XS	PFLAP-S	PFLAP-M	PFLAP-L	PFLAP-XL	PFLAP-XXL
(10" x 16") (26cm x 41cm)	(13" x 16") (33cm x 41cm)	(15" x 16") (38cm x 41cm)	(17" x 16") (43cm x 41cm)	(19" x 16") (48cm x 41cm)	(21" x 16") (53cm x 41cm)

## PFLAP Installation Instructions:

1. Remove the wheelchair seat cushion (if installed).
2. Align/attach the hook strip (A) on the PFLAP onto the loop strip (B) on the bottom rear of the Matrix Back (1).
3. Pull the opposite end of the privacy flap through the gap between the backrest and the wheelchair seat, and secure the loop strip on the PFLAP (C) to the wheelchair seat\* (2).

**\*NOTE:** For sling seats/seat pans that are configured with a loop strip, attach the **double-sided hook adapter (D)** to secure the PFLAP in place (3).

4. Re-install the wheelchair seat cushion.

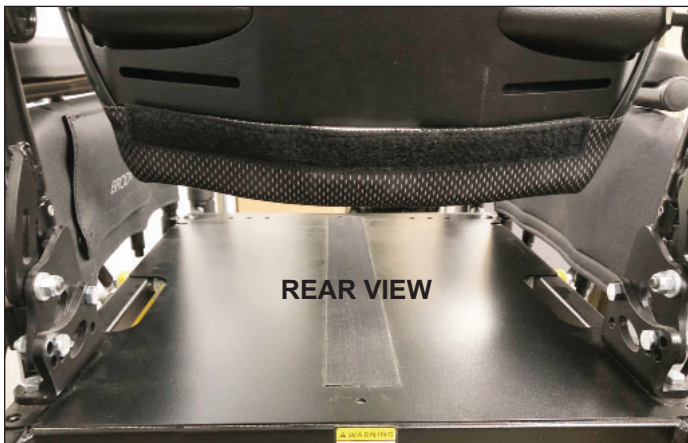

(note: Matrix Elite Back shown)





## IMPORTER:

France:  
 Invacare France Operations SAS  
 Route de St Roch  
 F-37230 Fondettes  
 Tel: (33) (0)2 47 62 64 66  
 contactfr@invacare.com;  
 www.invacare.fr

United Kingdom:  
 Invacare Limited  
 Pencoed Technology Park,  
 Pencoed Bridgend CF35 5AQ  
 Tel: (44) (0) 1656 776 200  
 uk@invacare.com;  
 www.invacare.co.uk



## DISTRIBUTOR:

Belgium & Luxemburg:  
 Invacare nv  
 Autobaan 22  
 B-8210 Loppem  
 Tel: (32) (0)50 83 10 10  
 marketingbelgium@invacare.com  
 www.invacare.be

France:  
 Invacare Poirier SAS  
 Route de St Roch  
 F-37230 Fondettes  
 Tel: (33) (0)2 47 62 64 66  
 contactfr@invacare.com;  
 www.invacare.fr

Norge:  
 Besøksadresse: (Office Address)  
 Invacare AS  
 Brynsveien 16  
 0667 Oslo  
 Tel: (47) (0)22 57 95 00  
 norway@invacare.com; www.  
 invacare.no

Suomi:  
 Camp Mobility  
 Patamäenkatu 5,  
 33900 Tampere  
 Puhelin 09-35076310  
 info@campmobility.fi  
 www.campmobility.fi

España:  
 Invacare SA  
 Invacare SA  
 Avenida del Oeste, 50-1º-1a  
 Tel: (34) (0)972 493 214  
 contactsp@invacare.com  
 www.invacare.es

Ireland:  
 Invacare Ireland Ltd,  
 Unit 5 Seatown Business Campus  
 Seatown Road, Swords,  
 County Dublin  
 Tel : (353) 1 810 7084  
 ireland@invacare.com  
 www.invacare.ie

Österreich:  
 Invacare Austria GmbH  
 Herzog Odilo-Strasse 101  
 A-5310 Mondsee  
 Tel: (43) 6232 5535 0  
 info-austria@invacare.com  
 www.invacare.at

Sverige:  
 Invacare AB  
 Fagerstagatan 9  
 S-163 53 Spånga  
 Tel: (46) (0)8 761 70 90  
 sweden@invacare.com  
 www.invacare.se

Danmark:  
 Invacare A/S  
 Sdr. Ringvej 37  
 DK-2605 Brøndby  
 Tel: (45) (0)36 90 00 00  
 denmark@invacare.com  
 www.invacare.dk

Italia:  
 Invacare Mecc San s.r.l.,  
 Via Marco Corner, 19,  
 I-36016 Thiene (VI)  
 Tel: (39) 0445 38 00 59  
 italia@invacare.com;  
 www.invacare.it

Portugal:  
 Invacare Lda  
 Rua Estrada Velha, 949  
 P-4465-784 Leça do Balio  
 Tel: (351) (0)225 1059 46/47  
 portugal@invacare.com  
 www.invacare.pt

United Kingdom:  
 Invacare Limited  
 Pencoed Technology Park,  
 Pencoed Bridgend CF35 5AQ  
 Tel: (44) (0) 1656 776 200  
 uk@invacare.com  
 www.invacare.co.uk

Deutschland:  
 Invacare GmbH  
 Am Achener Hof 8  
 D-88316 Isny  
 Tel: (49) (0)7562 700 0  
 kontakt@invacare.com; www.  
 invacare.de

Nederland:  
 Invacare BV  
 Galvanistraat 14-3  
 NL-6716 AE Ede  
 Tel: (31) (0)318 695 757  
 nederland@invacare.com  
 www.invacare.nl

Schweiz / Suisse / Svizzera:  
 Invacare AG  
 Benkenstrasse 260  
 CH-4108 Witterswil  
 Tel: (41) (0)61 487 70 80  
 switzerland@invacare.com  
 www.invacare.ch

USA:  
 Invacare America  
 Elyria, Ohio,  
 USA 44036-2125  
 Tel: (1) 800 333 6900  
 www.invacare.com



**Motion Concepts**  
 505 Cityview Blvd, Unit 1  
 Vaughan, Ontario, L4H 0L8 Canada  
 Toll Free Tel: (1) 866.748.7943  
 Tel: (1) 905.695.0134  
 www.motionconcepts.com



Advana Ltd.  
 Tower Business Center, 2nd Flr  
 Tower Street, Swatar BKR 4103 Malta

