



Chest Harness Interface

for the

Invacare-Matrix Mini Back

(model #: HIMINI)

Installation Manual

i DEALER: This manual contains important Safety and Installation information specific to Motion Concepts Chest Harness Interface for use on the IVC-Matrix Mini Back . This Manual MUST be given to the user of the product.

BEFORE using this product, please read through the instructions carefully and save for future reference.



SAFETY WARNINGS:



WARNING! The Chest Harness Interface for the Matrix Mini Back (HIMINI) is designed for use with approved OEM support/positioning harnesses (*not provided*).

- ALWAYS ensure all harness fittings and hardware is compatible with the chest harness interface brackets provided.
- DO NOT modify the interface mounting brackets; Modifications to the mounting brackets may result in personal injury and will compromise the limited warranty.



WARNING! Risk of serious personal injury or death.

Support Harnesses and Positioning Belts are not designed to provide proper restraint during motor vehicle transport and should never be utilized as a Safety Restraint.

- DO NOT sit in your wheelchair while it is in a moving vehicle. When travelling in a motor vehicle you must be properly secured in a motor vehicle seat with approved safety belts securely fastened.



WARNING! Risk of serious personal injury or death.

The HIMINI interface brackets have NOT been crash tested and are NOT approved for use as part of a vehicle safety restraint during transport.

- When travelling in a motor vehicle, the user must be properly secured in a motor vehicle seat with approved safety belts securely fastened.



WARNING! Risk of serious injury.

Please check your support/positioning harness weekly to ensure that it is still in good working condition; to ensure it has no damage or wear, and to ensure it is fixed properly to the wheelchair.

- If the support harness is fastened using a bolted connection, ensure that the connection has not loosened or come undone.
- Please be certain to read the service manual provided by the support/positioning harness manufacturer for more information about safety and maintenance of your harness.



IMPORTANT!

Please be certain to read the **User Manual** provided (separately) with your Positioning/Support Harness for important safety information and instructions regarding the proper set-up, installation and use of your harness.


If you have any questions or concerns during the installation of the (HIMINI) Chest Harness Interface, please contact our Technical Service Department for assistance:

Canada: 800-680-4191

USA: 888-433-6818

1.0 HIMINI BRACKET INSTALLATION

1.1 Chest Harness Bracket Installation Instructions:

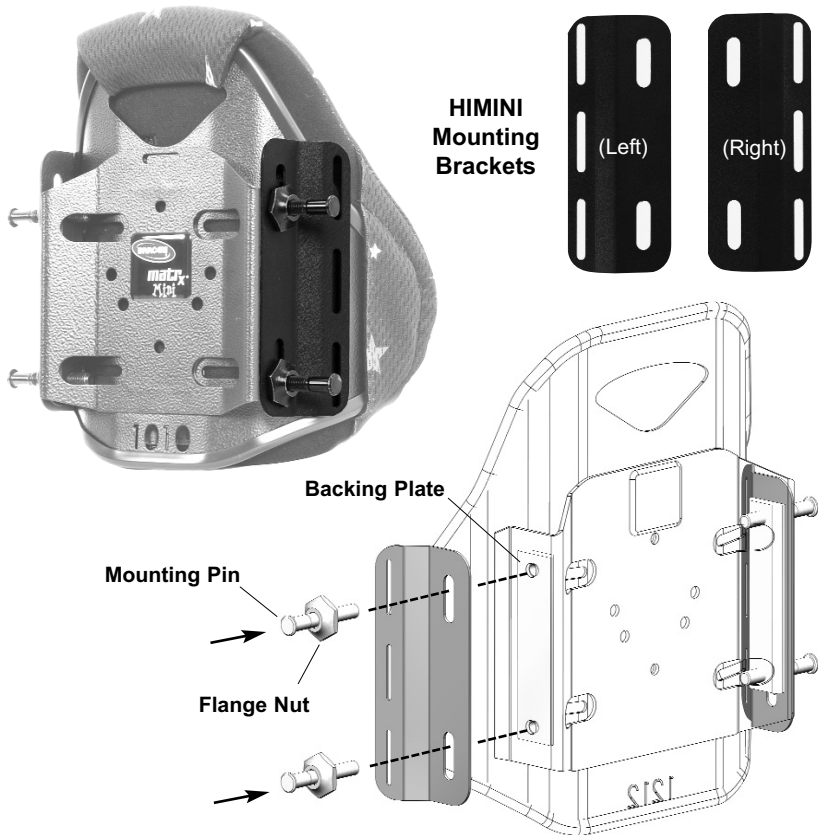

- 1 Working on one side of the Matrix Mini back shell, loosen and remove the mounting pins and flange nuts from the backing plate.
- 2 Align the double slots on HIMINI mounting bracket with the mounting holes in the Mini back shell, and secure the bracket into position using the mounting pins/flange nuts (*see images above*).
(Note: please refer to your Matrix Mini User Manual for instructions on setting the proper mounting pin depth).
IMPORTANT! Once the pin depth has been properly adjusted, ensure the flange nuts are fully tightened against the mounting bracket.
- 3 Repeat steps 1 and 2 for the opposite side of the back shell.
- 4 With both HIMINI mounting brackets installed, please be certain to read the **OEM User Manual** provided (*separately*) with the Support/Positioning Harness for important safety information and instructions regarding proper harness installation and set-up.

Contact Information
Informations de contact
Kontakttdaten
Información de contacto
Informações de contacto

United Kingdom • Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ •
Tel: (44) (0) 1656 776 222 •
Fax: (44) (0) 1656 776 220 • www.invacare.co.uk •
UK@invacare.com

Ireland • Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin-Ireland • Tel: (353) 1 810 7084 •
Fax: (353) 1 810 7085 • www.invacare.co.uk •
ireland@invacare.com

France • Invacare France Opération, Route de Saint Roch, 37230 Fondettes • Tel: (33) (0) 2 47 62 64 66 •
Fax: (33) (0) 2 47 42 12 24 • www.invacare.fr •
contactfr@invacare.com

Belgium & Luxemburg • Invacare nv, Autobaan 22, B-8210 Loppem, Brügge • Tel: (32) (0) 50 83 10 10 •
Fax: (32) (0) 50 83 10 11 • www.invacare.be •
belgium@invacare.com

Switzerland • Invacare AG, Benkenstrasse 260, CH-4108 Witterswil • Tel: (41) (0) 61 487 70 80 •
Fax: (41) (0) 61 487 70 81 •
switzerland@invacare.com

Deutschland • Invacare GmbH, Alemannenstraße 10, D-88316 Isny • Tel: (49) (0) 75 62 7 00 0 •
Fax: (49) (0) 75 62 7 00 66 •
www.invacare-aquatec.de • info@invacare-aquatec.com

Eastern Europe, Middle East & Africa • Invacare European Distributor Organisation, Kleisestraße 49, D-32457 Porta Westfalica •
Tel: (49) (0) 57 31 754 540 • Fax: (49) (0) 57 31 754 541 •
www.invacare.eu.com • edo@invacare.com

Portugal • Invacare Lda, Rua Estrada Velha, 949, P-4465-784, Leça do Balio •
Tel.: (351) (0)225 1059 46/47 •
Fax: (351) (0)225 1057 39 •
portugal@invacare.com

Motion Concepts:

Canada • Motion Concepts, 84 Citation Drive, Unit 1, Concord, Ontario, L4K 3C1 •
Tel: (905) 695-0134 • Fax: (905) 695-0138 •
www.motionconcepts.com

USA • Motion Concepts, 700 Ensminger Rd., Suite 112, Tonawanda, New York 14150 •
Tel: (716) 447-0050 • Fax: (716) 447-0030 •
info@motionconcepts.com

Per informazioni
Kontaktoplysninger
Kontaktinformation
Yhteystiedot

Norge • Invacare AS, Grenseshvingen 9, Postboks 6230, Etterstad, N-0603 Oslo • Tel: (47) (0) 22 57 95 00 •
Fax: (47) (0) 22 57 95 01 • norway@invacare.com •
island@invacare.com

Italia • Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI) • Tel: (39) 0445 38 00 59 •
Fax: (39) 0445 38 00 34 • www.invacare.it •
italia@invacare.com

España • Invacare SA, c/Areny s/n, Polígono Industrial de Celrà, E-17460 Celrà (Girona) •
Tel: (34) (0) 972 49 32 00 • Fax: (34) (0) 972 49 32 20 •
www.invacare.es • contactsp@invacare.com

Canada • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 •
Tel: (905) 890-8300 • Fax: (905) 890-5244 •
www.invacare.ca

Danmark • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00 •
Fax: (45) (0) 36 90 00 01 •
www.invacare.dk • denmark@invacare.com

Sverige & Suomi • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090 •
Fax: (46) (0) 8 761 8108 • www.invacare.se •
sweden@invacare.com

Nederland • Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede • Tel: (31) (0)318 695 757 •
Fax: (31) (0)318 695 758 • www.invacare.nl •
nederland@invacare.com • csede@invacare.com

Österreich • Invacare Austria GmbH, Herzog Odillostrasse 101, A-5310 Mondsee, Austria •
Tel: (43) (0)6232 5 53 50 •
Fax: (43) (0)6232 5 53 54 •
www.invacare.at • info@invacare-austria.com

matrx
SEATING SERIES



EC REP

Advena Ltd. Tower Business Center,
2nd Flr., Tower Street, Swatar,
BKR 4013 Malta

