



# Mini Swing Away Laterals

*for the*

# Invacare-Matrix Mini Back

(model #: SAHMINI)

## Installation Manual

**i DEALER:** This manual contains installation & adjustment instructions specific to Motion Concepts Mini Swing-Away Laterals. This Manual MUST be given to the user of the product.


**BEFORE** using this product, please read through the instructions carefully and save for future reference.



# Installation Overview: Mini Swing-Away Lateral (SAHMINI)

## Mini Lateral Installation Overview:

(Note: Left Lateral Assembly Shown)

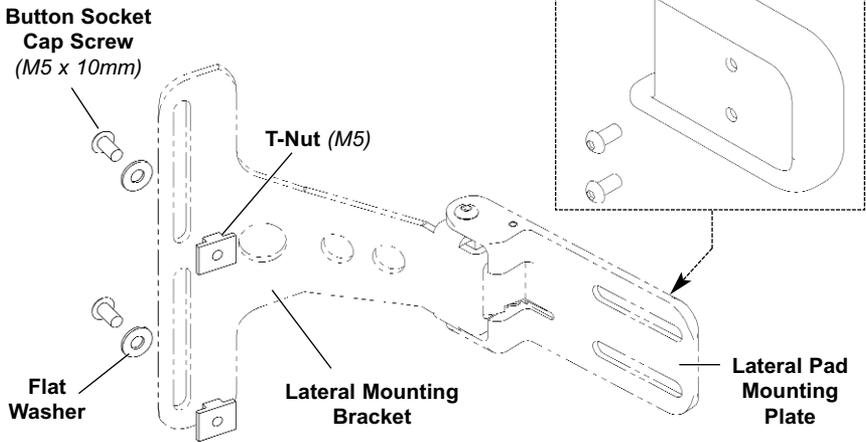

## Installation Instructions:

- 1 Prior to installation, the outer cover must be removed from the Matrix Mini back shell. To remove, disengage the bottom\* of the cover from the hook & loop fasteners and pull cover away from the back shell. (\*The upper portion of the cover can remain attached)
- 2 Working on one side of the back shell, align the vertical slots in the lateral mounting bracket with the horizontal slots in the back shell and *loosely* install the bracket onto the inside of the back shell using the hardware provided (see **Installation Overview** above).
- 3 Loosely install the remaining lateral mounting bracket on the opposite side of the back shell.
- 4 Install the Lateral Pads (x2) (*sold separately*) onto the lateral pad mounting plates using the hardware provided.
- 5 Adjust each lateral bracket to the desired position (width and height), on the back shell, then fully tighten the mounting hardware. (**note:** *The position of the lateral pad can also be adjusted to the desired depth/angle on each mounting plate*)
- 6 With both Mini Laterals properly installed/adjusted, re-install the outer cover onto the back shell.


*If you have any questions or concerns during the installation of the Mini Swing-Away Laterals, please contact our Technical Service Department for assistance:*

**Canada:** 800-680-4191

**USA:** 888-433-6818

# Mini Swing-Away Lateral (SAHMINI) Adjustments:

## Mini Lateral Adjustments:

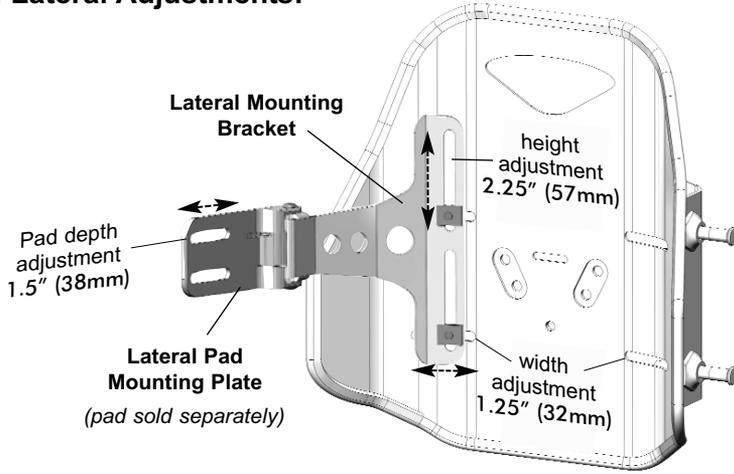

## Swing-Away Feature:



**Contact Information**  
**Informations de contact**  
**Kontaktdata**  
**Información de contacto**  
**Informações de contacto**

**United Kingdom** • Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ •  
Tel: (44) (0) 1656 776 222 •  
Fax: (44) (0) 1656 776 220 • www.invacare.co.uk •  
UK@invacare.com

**Ireland** • Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin-Ireland • Tel: (353) 1 810 7084 •  
Fax: (353) 1 810 7085 • www.invacare.co.uk •  
ireland@invacare.com

**France** • Invacare France Opération, Route de Saint Roch, 37230 Fondettes • Tel: (33) (0) 2 47 62 64 66 •  
Fax: (33) (0) 2 47 42 12 24 • www.invacare.fr •  
contactfr@invacare.com

**Belgium & Luxemburg** • Invacare nv, Autobaan 22, B-8210 Loppem, Brügge • Tel: (32) (0) 50 83 10 10 •  
Fax: (32) (0) 50 83 10 11 • www.invacare.be •  
belgium@invacare.com

**Switzerland** • Invacare AG, Benkenstrasse 260, CH-4108 Witterswil • Tel: (41) (0) 61 487 70 80 •  
Fax: (41) (0) 61 487 70 81 •  
switzerland@invacare.com

**Deutschland** • Invacare GmbH, Alemannenstraße 10, D-88316 Isny • Tel: (49) (0) 75 62 7 00 0 •  
Fax: (49) (0) 75 62 7 00 66 •  
www.invacare-aquatec.de • info@invacare-aquatec.com

**Eastern Europe, Middle East & Africa** • Invacare European Distributor Organisation, Kleisestraße 49, D-32457 Porta Westfalica •  
Tel: (49) (0) 57 31 754 540 • Fax: (49) (0) 57 31 754 541 •  
www.invacare.eu.com • edo@invacare.com

**Portugal** • Invacare Lda, Rua Estrada Velha, 949, P-4465-784, Leça do Balio •  
Tel.: (351) (0)225 1059 46/47 •  
Fax: (351) (0)225 1057 39 •  
portugal@invacare.com

**Motion Concepts:**

**Canada** • Motion Concepts, 84 Citation Drive, Unit 1, Concord, Ontario, L4K 3C1 •  
Tel: (905) 695-0134 • Fax: (905) 695-0138 •  
www.motionconcepts.com

**USA** • Motion Concepts, 700 Ensminger Rd., Suite 112, Tonawanda, New York 14150 •  
Tel: (716) 447-0050 • Fax: (716) 447-0030 •  
info@motionconcepts.com

**Per informazioni**  
**Kontaktoplysninger**  
**Kontaktinformation**  
**Yhteystiedot**

**Norge** • Invacare AS, Grensvesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo • Tel: (47) (0) 22 57 95 00 •  
Fax: (47) (0) 22 57 95 01 • norway@invacare.com •  
island@invacare.com

**Italia** • Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI) • Tel: (39) 0445 38 00 59 •  
Fax: (39) 0445 38 00 34 • www.invacare.it •  
italia@invacare.com

**España** • Invacare SA, c/Areny s/n, Poligon Industrial de Celrà, E-17460 Celrà (Girona) •  
Tel: (34) (0) 972 49 32 00 • Fax: (34) (0) 972 49 32 20 •  
www.invacare.es • contactsp@invacare.com

**Canada** • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 •  
Tel: (905) 890-8300 • Fax: (905) 890-5244 •  
www.invacare.ca

**Danmark** • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00 •  
Fax: (45) (0) 36 90 00 01 •  
www.invacare.dk • denmark@invacare.com

**Sverige & Suomi** • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090 •  
Fax: (46) (0) 8 761 8108 • www.invacare.se •  
sweden@invacare.com

**Nederland** • Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede • Tel: (31) (0)318 695 757 •  
Fax: (31) (0)318 695 758 • www.invacare.nl •  
nederland@invacare.com • csede@invacare.com

**Österreich** • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria •  
Tel: (43) (0)6232 5 53 50 •  
Fax: (43) (0)6232 5 53 54 •  
www.invacare.at • info@invacare-austria.com

**matrx**  
SEATING SERIES



**EC REP**

Advena Ltd. Tower Business Center,  
2nd Fl., Tower Street, Swatar,  
BKR 4013 Malta

