


## E2 HEADREST INTERFACE PLATE FOR ELITE E2 BACKS

The E2 Headrest Interface Plate is compatible with most industry standard headrest mounting hardware and is available in two models:


**E2HI** (Standard): The Standard E2 Headrest Interface Plate (A) is included with Elite E2S/E2D backs for **Back Heights** ranging from **14" (36cm)** up to **20" (51cm)**.

**E2HDI** (Heavy-Duty): The Heavy-Duty E2 Headrest Interface Plate offers an extra strong steel headrest mount for extreme tone or heavy use applications.

### E2 HEADREST INTERFACE PLATE INSTALLATION

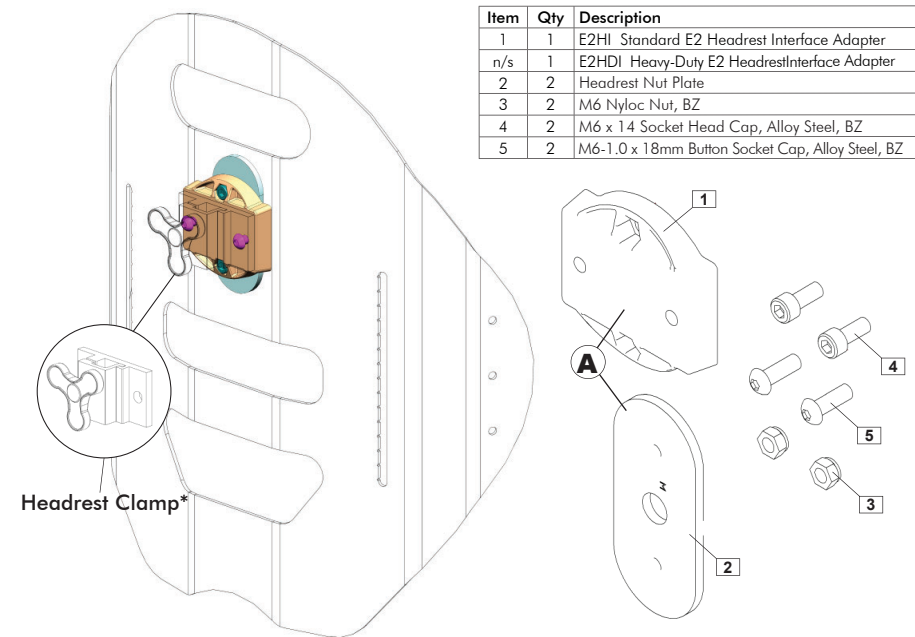

**\*NOTE:** The Headrest Clamp assembly (sold separately) must be installed onto the E2 Headrest Interface Plate (1) PRIOR to installation onto the back shell (2)

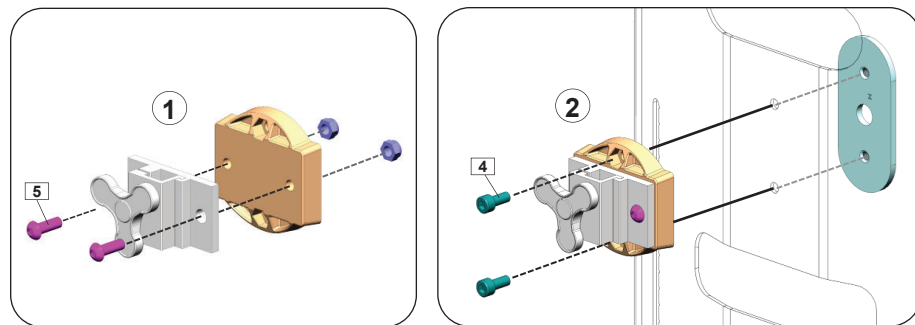

TRD0582 Rev B

### Invacare Distributors:

Belgium & Luxembourg:  
Invacare nv Autobaan 22  
B-8210 Loppem  
Tel: (32) (0)50 83 10 10  
Fax: (32) (0)50 83 10 11  
belgium@invacare.com  
www.invacare.be

España:  
Invacare SA  
c/Areny s/n, Polígon Industrial  
de Celrà  
E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00  
Fax: (34) (0)972 49 32 20  
contactsp@invacare.com  
www.invacare.es

Danmark:  
Invacare A/S Sdr. Ringvej 37  
DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00  
Fax: (45) (0)36 90 00 01  
denmark@invacare.com  
www.invacare.dk

France:  
Invacare Poirier SAS  
Route de St Roch  
F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66  
Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com  
www.invacare.fr

Portugal:  
Invacare Lda  
Rua Estrada Velha, 949  
P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47  
Fax: (351) (0)225 1057 39  
portugal@invacare.com  
www.invacare.pt

USA:  
Invacare USA  
Worldwide Headquarters  
One Invacare Way  
Elyria, Ohio,  
USA 44036-2125  
Tel: (1) 800 333 6900  
Fax: (1) 877 619 7996  
www.invacare.com

Deutschland:  
Invacare GmbH  
Alemannenstraße 10  
D-88316 Isny  
Tel: (49) (0)7562 700 0  
Fax: (49) (0)7562 700 66  
kontakt@invacare.com  
www.invacare.de

Ireland:  
Invacare Ireland Ltd,  
Unit 5 Seatown Business Campus  
Seatown Road, Swords,  
County Dublin  
Tel: (353) 1 810 7084  
Fax: (353) 1 810 7085  
ireland@invacare.com  
www.invacare.ie

Eastern Europe, Middle East & CIS:  
Invacare EU Export Kleiststraße 49  
D-32457 Porta Westfalica /  
Germany  
Tel: (49) (0)57 31 754 540  
Fax: (49) (0)57 31 754 541  
webinfo-eu-export@invacare.com  
www.invacare-eu-export.com

Italia:  
Invacare Mecc San s.r.l.,  
Via dei Pini 62,  
I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59  
Fax: (39) 0445 38 00 34  
italia@invacare.com  
www.invacare.it

United Kingdom:  
Invacare Limited  
Pencoed Technology Park,  
Pencoed  
Bridgend CF35 5AQ  
Tel: (44) (0) 1656 776 222  
Fax: (44) (0) 1656 776 220  
uk@invacare.com  
www.invacare.co.uk

Canada:  
Invacare Canada  
570 Matheson Blvd. E. Unit 8  
Mississauga, Ontario  
Canada L4Z 4G4  
Tel: (800) 668 5324  
Fax: (905) 890 5244  
www.invacare.ca

Nederland:  
Invacare BV Galvanistraat 14-3  
NL-6716 AE Ede  
Tel: (31) (0)318 695 757  
Fax: (31) (0)318 695 758  
nederland@invacare.com  
www.invacare.nl

Suomi:  
Camp Mobility  
Patamäenkatu 5,  
33900 Tampere  
Puhelin 09-35076310  
info@campmobility.fi  
www.campmobility.fi

Norge:  
Invacare AS  
Grensesvingen 9, Postboks 6230,  
Etterstad  
N-0603 Oslo  
Tel: (47) (0)22 57 95 00  
Fax: (47) (0)22 57 95 01  
norway@invacare.com  
www.invacare.no

Sverige:  
Invacare AB Fagerstagatan 9  
S-163 53 Spånå  
Tel: (46) (0)8 761 70 90  
Fax: (46) (0)8 761 81 08  
sweden@invacare.com  
www.invacare.se

Österreich:  
Invacare Austria GmbH Herzog Odi-  
lostrasse 101  
A-5310 Mondsee  
Tel: (43) 6232 5535 0  
Fax: (43) 6232 5535 4  
info-austria@invacare.com  
www.invacare.at

Schweiz / Suisse / Svizzera:  
Invacare AG Benkenstrasse 260  
CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80  
Fax: (41) (0)61 487 70 81  
switzerland@invacare.com  
www.invacare.ch



Motion Concepts  
84 Citation Drive, Unit 1  
Concord, Ontario L4K 3C1  
Canada



Advena Ltd.  
Tower Business Cebter, 2nd Flr  
Tower Street, Swatar BKR 4103  
Malta




## E2 HEADREST INTERFACE PLATE FOR ELITE E2 BACKS

The E2 Headrest Interface Plate is compatible with most industry standard headrest mounting hardware and is available in two models:


**E2HI** (Standard): The Standard E2 Headrest Interface Plate (A) is included with Elite E2S/E2D backs for **Back Heights** ranging from **14" (36cm)** up to **20" (51cm)**.

**E2HDI** (Heavy-Duty): The Heavy-Duty E2 Headrest Interface Plate offers an extra strong steel headrest mount for extreme tone or heavy use applications.

### E2 HEADREST INTERFACE PLATE INSTALLATION

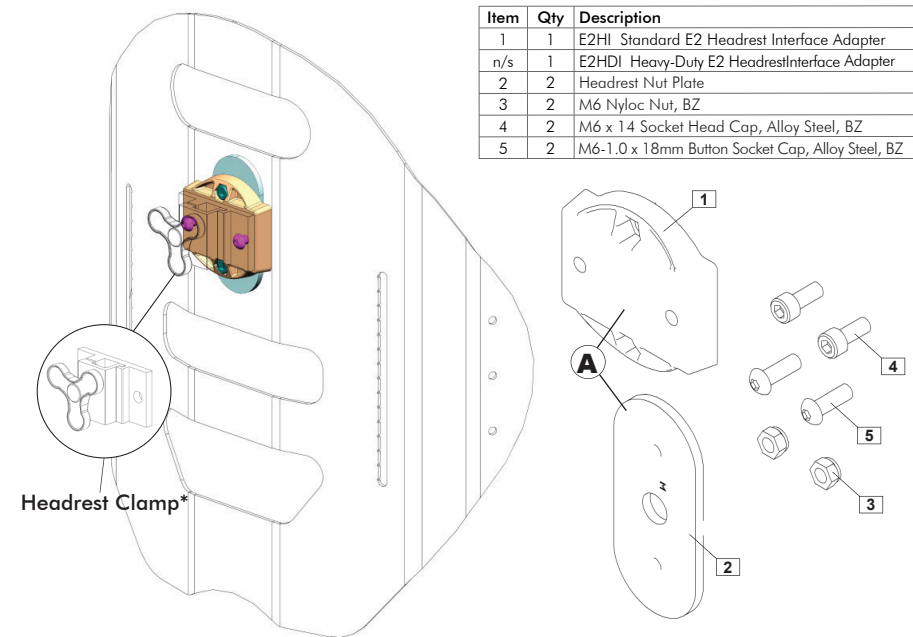

**\*NOTE:** The Headrest Clamp assembly (sold separately) must be installed onto the E2 Headrest Interface Plate (1) **PRIOR** to installation onto the back shell (2)

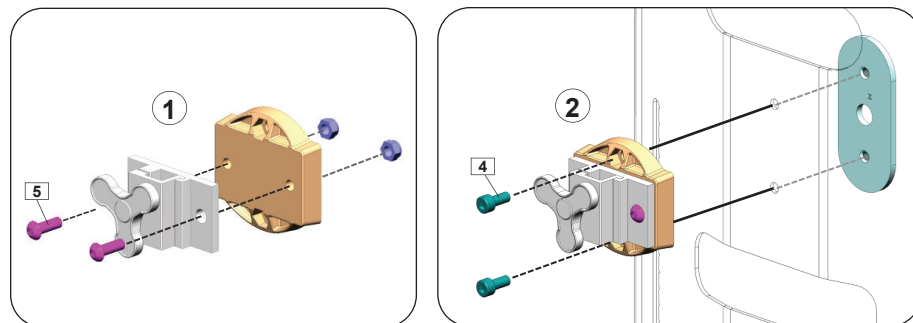

TRD0582 Rev B

### Invacare Distributors:

Belgium & Luxembourg:  
Invacare nv Autobaan 22  
B-8210 Loppem  
Tel: (32) (0)50 83 10 10  
Fax: (32) (0)50 83 10 11  
belgium@invacare.com  
www.invacare.be

España:  
Invacare SA  
c/Areny s/n, Polígon Industrial  
de Celrà  
E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00  
Fax: (34) (0)972 49 32 20  
contactsp@invacare.com  
www.invacare.es

Danmark:  
Invacare A/S Sdr. Ringvej 37  
DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00  
Fax: (45) (0)36 90 00 01  
denmark@invacare.com  
www.invacare.dk

France:  
Invacare Poirier SAS  
Route de St Roch  
F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66  
Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com  
www.invacare.fr

Portugal:  
Invacare Lda  
Rua Estrada Velha, 949  
P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47  
Fax: (351) (0)225 1057 39  
portugal@invacare.com  
www.invacare.pt

USA:  
Invacare USA  
Worldwide Headquarters  
One Invacare Way  
Elyria, Ohio,  
USA 44036-2125  
Tel: (1) 800 333 6900  
Fax: (1) 877 619 7996  
www.invacare.com

Deutschland:  
Invacare GmbH  
Alemannenstraße 10  
D-88316 Isny  
Tel: (49) (0)7562 700 0  
Fax: (49) (0)7562 700 66  
kontakt@invacare.com  
www.invacare.de

Ireland:  
Invacare Ireland Ltd,  
Unit 5 Seatown Business Campus  
Seatown Road, Swords,  
County Dublin  
Tel: (353) 1 810 7084  
Fax: (353) 1 810 7085  
ireland@invacare.com  
www.invacare.ie

Eastern Europe, Middle East & CIS:  
Invacare EU Export Kleiststraße 49  
D-32457 Porta Westfalica /  
Germany  
Tel: (49) (0)57 31 754 540  
Fax: (49) (0)57 31 754 541  
webinfo-eu-export@invacare.com  
www.invacare-eu-export.com

Italia:  
Invacare Mecc San s.r.l.,  
Via dei Pini 62,  
I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59  
Fax: (39) 0445 38 00 34  
italia@invacare.com  
www.invacare.it

United Kingdom:  
Invacare Limited  
Pencoed Technology Park,  
Pencoed  
Bridgend CF35 5AQ  
Tel: (44) (0) 1656 776 222  
Fax: (44) (0) 1656 776 220  
uk@invacare.com  
www.invacare.co.uk

Canada:  
Invacare Canada  
570 Matheson Blvd. E. Unit 8  
Mississauga, Ontario  
Canada L4Z 4G4  
Tel: (800) 668 5324  
Fax: (905) 890 5244  
www.invacare.ca

Nederland:  
Invacare BV Galvanistraat 14-3  
NL-6716 AE Ede  
Tel: (31) (0)318 695 757  
Fax: (31) (0)318 695 758  
nederland@invacare.com  
www.invacare.nl

Suomi:  
Camp Mobility  
Patamäenkatu 5,  
33900 Tampere  
Puhelin 09-35076310  
info@campmobility.fi  
www.campmobility.fi

Norge:  
Invacare AS  
Grensesvingen 9, Postboks 6230,  
Etterstad  
N-0603 Oslo  
Tel: (47) (0)22 57 95 00  
Fax: (47) (0)22 57 95 01  
norway@invacare.com  
www.invacare.no

Sverige:  
Invacare AB Fagerstagatan 9  
S-163 53 Spånå  
Tel: (46) (0)8 761 70 90  
Fax: (46) (0)8 761 81 08  
sweden@invacare.com  
www.invacare.se

Österreich:  
Invacare Austria GmbH Herzog Odi-  
lostrasse 101  
A-5310 Mondsee  
Tel: (43) 6232 5535 0  
Fax: (43) 6232 5535 4  
info-austria@invacare.com  
www.invacare.at

Schweiz / Suisse / Svizzera:  
Invacare AG Benkenstrasse 260  
CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80  
Fax: (41) (0)61 487 70 81  
switzerland@invacare.com  
www.invacare.ch



Motion Concepts  
84 Citation Drive, Unit 1  
Concord, Ontario L4K 3C1  
Canada

**EC REP**

Advena Ltd.  
Tower Business Cebter, 2nd Flr  
Tower Street, Swatar BKR 4103  
Malta

